



X Series[®] Configuration Manual



Includes Real CPR Help[®] See-Thru CPR[®]

The issue date for the Configuration Manual (REF 9650-001358-01 Rev. M) is October, 2023.

Copyright © 2023 ZOLL Medical Corporation. All rights reserved. AutoPulse, RapidShock, Real CPR Help, RescueNet, See-Thru CPR, X Series, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.

Masimo, Rainbow, SET, SpCO, SpMet, SpHb, SpOC, and PVI are trademarks or registered trademarks of Masimo Corporation in the United States and/or other countries.

This product and accompanying components are manufactured and sold under one or more patents listed at the following internet address:

<http://www.zoll.com/patents>



ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA USA
01824-4105



ZOLL International Holding B.V.
Einsteinweg 8A
6662 PW Elst
Netherlands



0123

CONTENTS

CHAPTER 1 Introduction

Overview	2
Setup Menu	3
Parameter Menus	3
Supervisor Menu	4
Exporting and Importing Configuration Settings	5
Export/Import System Messages	6
Restoring Factory Default Configuration Settings	8
Restoring Default Values	8
Configuration Manual Updates	9
Symbols	9
Conventions	10

CHAPTER 2 Setup Menu

Accessing the Setup Menu	12
ECG Monitoring Parameters	13
Filter Response	13
HR/PR Tone	13
Resp Monitoring	14
Resp Lead	14
Patient Pace Marker	14
ECG Sweep Speed	14
NIBP Monitoring Parameters	15
NIBP Cuff Inflation Target	15
Display/Volume Parameters	16
Display Brightness	16
HR/PR Tone	16

Volume	16
Printer Parameters	17
Print Speed	17
Print Number of Traces	17
Print Grid	17
Print on Snapshot	18
Print on Patient Alarm	18
Print on NIBP	18
Print on Trend	18
Trends Parameters	19
Trend Display Format	19
Trend Display Interval	19
Trend on Alarm	19
Operational Tests	21
Device Info	22
Readiness Test Log Review	22

CHAPTER 3 Parameter Menus

Accessing the Parameter Menus	24
HR/PR Parameter Settings	25
HR/PR Alarm	25
Extreme Brady/Tachy Alarms	26
LTA Monitoring	26
HR/PR Tone	27
HR/PR Selected Source	27
ECG Sweep Speed	27
NIBP Parameter Settings	28
NIBP Systolic Alarm	28
NIBP Diastolic Alarm	30
NIBP MAP Alarm	31
NIBP Mode	32
NIBP Auto Mode Interval	32
SmartCuf On/Off	33
Resp Parameter Settings	34
RR/BR Alarm	34
No Breath Alarm	35
Resp Monitoring	35
Resp Lead	37
CO2/Resp Sweep Speed	37

SpO2 Parameter Settings	38
SpO2 Alarm	39
SpCO Alarm	40
SpMet Alarm	41
SpHb Alarm	43
SpOC Alarm	45
PVI Alarm	46
PI Alarm	47
SpCO Monitoring	48
SpMet Monitoring	48
SpHb Monitoring	48
SpOC Monitoring	49
PVI Monitoring	49
PI Monitoring	49
Averaging Time	49
Sensitivity	49
SpHb Venous Mode	50
SpHb Averaging Time	50
HR/PR Tone	50
Temp Parameter Settings	51
T1 Alarm	51
T2 Alarm	52
Δ T Alarm	53
T1 Source Label	54
T2 Source Label	54
IBP1 Parameter Settings	55
P1 Systolic Alarm	55
P1 Diastolic Alarm	56
P1 Mean Alarm	57
Source Label	58
Display Format	58
IBP2 Parameter Settings	60
P2 Systolic Alarm	60
P2 Diastolic Alarm	61
P2 Mean Alarm	62
Source Label	63
Display Format	64
IBP3 Parameter Settings	65
P3 Systolic Alarm	65

P3 Diastolic Alarm	66
P3 Mean Alarm	67
Source Label	69
Display Format	69
CHAPTER 4 Supervisor Menu	
Accessing the Supervisor Menu	71
Alarms Parameters	72
General	72
Default Adult	73
Default Pediatric	96
Default Neonate	118
Log Parameters	140
Treatment Options	140
Waveform Recording	143
Defib/Pacer Parameters	144
Default Settings	144
Energy Escalation	147
ECG Parameters	149
General	149
12-Lead ECG	149
NIBP Parameters	153
Default NIBP Mode	153
Default Auto Mode Interval	153
Default Cuff Inflation Target (Adult)	153
Default Cuff Inflation Target (Ped.)	154
Default Cuff Inflation Target (Neo.)	154
Printer Parameters	155
Print on Defib/Pacer Events	155
Print on Event	155
Print on Treatment Snapshot	155
Print Treatment Summary Snapshots	156
Print on Analysis Results	157
Print on Presenting Rhythm	157
Print On Readiness Test Results	157
Display/Configuration Parameters	158
Localization	158
Color	162
Device Info	167

Display Setup	169
Supervisor Code	171
Log Clear Code	172
AED/Advisory Parameters	173
General	173
Energy Protocol	178
AED Mode	180
CPR Parameters	182
AHA/ERC Guidelines	182
CPR Time	182
Enable Metronome	183
Constant Metronome	185
Enable CPR Prompts	186
Display “Fully Release” Prompt	186
Log CPR Metrics	186
Auto Display Compression Trace	186
ResQCPR™	187
Communications Parameters	188
Wireless	190
Bluetooth	190
Wi-Fi	190
Cellular	195
Ethernet	197
12-Lead Reports	198
Disclosure Log	200
Data Services	201
Clock Sync	204
Mobile Streaming	207
Readiness Test	208
Service Parameters	210
Device Tests	210
Device Configuration	210
Update/Upgrade	210
Diagnostic	210
Device Monitoring	212
Research Options	212
Service Code	212

APPENDIX A Configuration Tables

Setup Menu 214
Parameter Menus 217
Supervisor Menu 234

CHAPTER 1

Introduction


This chapter provides an overview of the Setup, Parameter, and Supervisor Menus.

It includes the following sections:


Overview	2
Restoring Factory Default Configuration Settings	8
Configuration Manual Updates	9

Overview

The X Series® unit has many user accessible settings that allow you to configure the device according to your protocols and local requirements. This manual describes how to view and modify configuration settings.

 **Note:** Your unit only contains the features you have purchased. It may not contain all of the features listed in this manual.

You can view and modify configuration settings through the **Setup**, **Parameter**, and **Supervisor** menus. The Setup and Parameter menus are available to all users, while the Supervisor menu is restricted to authorized users and requires a passcode for entry. The Setup menu allows you to change configuration settings such as display, printer, and trends. The Parameter menus allow you to change current configuration settings such as NIBP and SpO₂. The Supervisor menu allows you to configure default settings such as alarms, NIBP, and Defib/Pacer.

 **Note:** The X Series unit ships from ZOLL® with both the default supervisor passcode and the service passcode set to 1234. ZOLL strongly recommends changing both passcodes when you receive the unit.

The Parameter menus share many of the same settings as the Supervisor menu. If the shared settings have been configured in the Supervisor menu, they can be modified in the Parameter menus, but they will default back to the Supervisor settings once the unit is turned off for two minutes.

The Setup, Parameter, and Supervisor menus are described in detail in separate chapters of this manual. These chapters contain information on how to access the menus as well as the settings that are contained in each menu and their default values.

The following sections contain a general overview of the settings that can be accessed and modified in these menus. Each of these menus has a separate chapter in this manual that includes all the settings and their default values.

Setup Menu

The Setup menu allows you to view and modify parameters for the following features:

- **ECG** – The basic ECG settings, which include Filter Response, Heart Rate/Pulse Rate Tone, and Respiration Monitoring.
- **NIBP** – The basic NIBP settings, which consist of the NIBP Cuff Inflation Target Pressure.
- **Display/Volume** – The basic visual settings and volume, which include Display Brightness and Heart Rate/Pulse Rate Tone.
- **Printer** – The basic printer settings, which include Print Speed, Print Grid, and Print on Snapshot.
- **Trends** – The basic trend settings, which include Trend Display Format, logging trends in response to Interval, and logging trends in response to Alarms.
- **Operational Tests** – The operational tests, which include a Keypad Test, LED Test, and LCD Test.
- **Device Info** – The technical information, which includes Application Version, Serial Number, and Unit ID. In addition, from the Device Info menu, you can transmit or print the results of the most recent device readiness test.

Parameter Menus

The Parameter menus allow you to view and modify parameters for the following features:

- **HR/PR Settings** – The heart monitoring protocols, which include HR/PR Alarm, LTA Monitor, and Extreme Brady/Tachy Alarm.
- **NIBP Settings** – The noninvasive blood pressure monitoring protocols, which include NIBP Systolic Alarm, NIBP Diastolic Alarm, NIBP MAP Alarm.
- **Resp Settings** – The respiration monitoring protocols, which include RR/BR Alarm, No Breath Alarm, and Respiration Monitoring.
- **SpO2 Settings** – The pulse CO-oximetry monitoring protocols, which include SpO2 Alarm and HR/PR Tone.
- **Temp Settings** – The temperature monitoring protocols, which include the alarm for channel 1, the alarm for channel 2, and the alarm for the difference between the two channels.
- **IBP1 Settings** – The invasive blood pressure monitoring protocols for the first channel, which include Systolic Pressure Alarm, Diastolic Pressure Alarm, and Mean Pressure Alarm.
- **IBP2 Settings** – The invasive blood pressure monitoring protocols for the second channel, which include Systolic Pressure Alarm, Diastolic Pressure Alarm, and Mean Pressure Alarm.
- **IBP3 Settings** – The invasive blood pressure monitoring protocols for the third channel, which include Systolic Pressure Alarm, Diastolic Pressure Alarm, and Mean Pressure Alarm.

Supervisor Menu

The Supervisor menu allows you to view and modify the default technical settings of the unit. Access to the Supervisor menu is passcode-protected and allows only authorized users to view and modify the configuration settings. The Supervisor menu allows authorized users to view and modify the following features:

- **Alarms Settings**
Enable or disable alarms on the unit and specify the alarm limits.
- **Log Settings**
The Data recording settings, which includes Treatment Buttons and Waveform Recording.
- **Defib/Pacer Settings**
The defibrillation and pacer settings, which include external and internal paddle energy settings and Default Pacer Mode.
- **ECG Settings**
The ECG settings, which include AC Mains Filter and 12-Lead ECG.
- **NIBP Settings**
NIBP settings include Default NIBP Mode, Default Auto Mode Interval, and default cuff inflation target pressures.
- **Printer Settings**
Printer settings include Print on Defib/Pacer Events, Print on Event, and Print on Treatment Snapshot.
- **Display/Configuration Settings**
Display settings include language, unit settings, and color settings.
- **AED Settings**
The AED settings, which include voice prompts, CPR time, and energy protocols
- **CPR Settings**
The CPR settings, which include CPR Time, Enable Metronome, and Enable CPR Prompts.
- **Communications Settings**
Communications settings include Wireless, Bluetooth, and Wi-Fi communication.

- **Ventilation Feedback**
- **Service Settings**

Technical maintenance settings include Device Tests, software updates, upgrades, diagnostics.

Exporting and Importing Configuration Settings

You can use a USB device to export and import a configuration file that contains system and patient setup information. Once a configuration file is exported to a USB device, it can be imported to other X Series units.



If you are planning on exporting a configuration to multiple units, perform the export operation with a unit that has the most features or options. Otherwise, you may inadvertently modify configuration settings on the units that receive the imported file. For example, if you export a configuration file from a unit that does not have SpO₂ or NIBP options and import that file to a unit that does have SpO₂ and NIBP, the configuration settings for those options may revert to the default settings.


Exporting Configuration Information to USB


Exporting a configuration file exports the system and patient setup information to a file named ZollConfig.xml.

 **Note:** If you currently have a file named ZollConfig.xml on the USB device, this procedure will cause the file to be overwritten.

Follow these steps to export a configuration to a USB device:


1. Insert the USB device into the X Series unit.
2. Press the **More**  and **Setup**  quick access keys, and select **Supervisor>Display/Configuration**.
3. Select **Exporting to USB**. The message *DO NOT DISCONNECT THE USB* displays.
4. When the export is complete, the message *EXPORT IS COMPLETE. YOU MAY NOW REMOVE THE USB DRIVE* displays.
5. Press **OK**.

 **Caution:** Apply an adhesive label to the USB device to clearly identify the configuration settings you have exported.



 **Note:** If you make any changes to the configuration file you have just exported, you will not be able to import the file to another X Series unit.


Importing Configuration Information from USB

Importing a configuration file imports the system and patient setup information from a file named ZollConfig.xml on the USB device.

 **Note:** Make sure the file you are importing is named ZollConfig.xml. Otherwise, the import will not work.

Follow these steps to import a configuration from a USB device:

1. Insert the USB device into the X Series unit, and turn it on.
2. Press the **More** () and **Setup** () quick access keys, and select **Supervisor>Display/Configuration**.
3. Select **Importing from USB**. The message *OPERATION WILL OVERWRITE SYSTEM CONFIGURATIONS AND ALARM SETTINGS. ARE YOU SURE?* displays.
4. Press **YES** to complete the import process. The message *DO NOT DISCONNECT THE USB* displays. When the import is complete, the message *IMPORT IS COMPLETE. YOU MAY NOW REMOVE THE USB DRIVE. THE DEVICE WILL AUTOMATICALLY RESET AFTER PRESSING THE "OK" BUTTON* displays.
5. Press **OK**.

 **Note:** Review critical configuration values such as alarm, AED, defibrillator, and pacemaker settings to verify that you imported the correct configuration file.


Export/Import System Messages

When exporting and importing system and patient information, the X Series unit may display the following error messages:


System Message	Cause/Action
<i>CONFIGURATION FILE NOT FOUND</i>	The configuration file (ZollConfig.xml) was not found on the USB device. Ensure that the connected USB device contains the configuration file. If required, generate a configuration file using the Export operation.
<i>EXPORT OPERATION FAILED</i>	An internal error occurred during the export process. Try repeating the export operation. If failure occurs again, try a different USB drive.
<i>IMPORT OPERATION FAILED</i> <i>INVALID CONFIGURATION FILE</i>	If the USB device is not new, retry the export and then retry the import operation. If the USB device is new and has not been formatted, reformat the device by selecting the FAT32 File System, changing the Allocation unit size to Default, and choosing the Quick Format option. Retry import operation.
<i>IMPORT OPERATION FAILED</i> <i>A RESTORE ALL TO FACTORY DEFAULTS HAS BEEN PERFORMED</i>	During the import process an unexpected failure occurred. All system and patient parameters have been restored to factory defaults. Retry import operation.
<i>INCOMPATIBLE CONFIGURATION FILE</i>	The device that exported the configuration file is not compatible with the device that is importing the file. This is caused by incompatible software releases. Contact the ZOLL Technical Service Department to update the earlier software version.
<i>INSUFFICIENT SPACE ON USB DRIVE</i>	The USB device did not have enough space to save the configuration file. The configuration file requires at least 1MB of free space.
<i>INVALID USB DRIVE CONNECTED</i>	The USB device connected was not recognized. Try a different USB device.
<i>NO USB DRIVE INSERTED</i>	No USB device was inserted in the side panel. Insert USB drive and retry.

Restoring Factory Default Configuration Settings

You can restore all the current configuration settings to their factory default settings. However, if the language is different than the default setting, it will not change.

 **Note:** You must have authorized supervisor access to restore the factory default settings.

Restoring Default Values

1. Press the **More**  and **Setup**  quick access keys, and select **Supervisor**.
2. Enter the Supervisor passcode and select **Display/Configuration**.
3. Use the navigation keys to select **Restore All Factory Defaults**.

Configuration Manual Updates







An issue or revision date for this manual is shown on the front cover. If more than three years have elapsed since this date, contact ZOLL Medical Corporation to determine if additional product information updates are available.

All users should carefully review each manual update to understand its significance and then file it in its appropriate section within this manual for subsequent reference.

Product documentation is available through the ZOLL website at www.zoll.com. From the Products menu, choose **Product Manuals**.

Symbols

The following symbols are used in this manual:


Symbol	Description
	Attention, consult accompanying documents
	Manufacturer.
	Authorized representative in the European Community
	Catalogue number
	Indicates the entity importing the medical device into the locale.
	Conformité Européenne Complies with medical device directive 93/42/EEC.


Conventions

This guide uses the following conventions:

Within text, the names and labels for physical buttons and soft keys appear in **boldface** type (for example, “Press the **OK** button”).

This guide uses uppercase italics for audible prompts and for text messages displayed on the screen (for example, *CONFIGURATION FILE NOT FOUND*).

 **WARNING!** Warning statements alert you to conditions or actions that can result in personal injury or death.

 **Caution:** Caution statements alert you to conditions or actions that can result in damage to the unit.

CHAPTER 2



Setup Menu

This chapter describes the configuration settings you can modify in the Setup menu.

It includes the following sections:

Accessing the Setup Menu	12
ECG Monitoring Parameters	13
NIBP Monitoring Parameters	15
Display/Volume Parameters	16
Printer Parameters	17
Trends Parameters	19
Operational Tests	21
Device Info	22

Accessing the Setup Menu

All users can access the Setup menu, which allows you to view and modify basic settings of the unit. Access the Setup menu by pressing the **More**  and **Setup**  quick access keys. Use the navigation keys to highlight and select the appropriate parameters in the menu.

The Setup Menu includes the following parameters:

- "ECG Monitoring Parameters" on the facing page
- "NIBP Monitoring Parameters" on page 15
- "Display/Volume Parameters" on page 16
- "Printer Parameters" on page 17
- "Trends Parameters" on page 19
- "Operational Tests" on page 21
- "Device Info" on page 22

ECG Monitoring Parameters

ECG monitoring parameters allow you to view and modify functions related to using the unit's ECG monitoring function. These parameters include:

- Filter Response
- HR/PR Tone
- Resp Monitoring
- Resp Lead
- Patient Pace Marker
- Patient Pace Speed
- ECG Sweep Speed

Filter Response

The Filter Response parameter allows you to specify the ECG bandwidth filter.

The values for the Filter Response parameter are:

Possible Values	Limited, Monitor
Default	Monitor

Limited: 0.67-20 Hz

Monitor: 0.67-40 Hz

HR/PR Tone

The HR/PR Tone parameter allows you to enable or disable the heart rate and pulse rate tone on the unit.

The values for the HR/PR Tone parameter are:

Possible Values	On, Off
Default	On

Resp Monitoring

The Resp Monitoring parameter allows you to enable or disable the respiration monitoring program.

The values for the Resp Monitoring parameter are:

Possible Values	On, Off
Default	On

Resp Lead

The Resp Lead parameter allows you to set which electrode the unit monitors as its respiration lead.

The values for the Resp Lead parameter are:


Possible Values	Lead I (RA-LA), Lead II (RA-LL)
Default	Lead II (RA-LL)

Patient Pace Marker

The Patient Pace Marker parameter allows you to enable or disable the patient Pace Marker.

The values for the Patient Pace Marker parameter are:

Possible Values	On, Off
Default	On

When you select **Off**, the Pacer Off Indicator () icon appears in the top ECG trace on the Display Screen to indicate the pacemaker indicator is off. If the patient has a cardiac pacemaker the Paced Indicator should be set to **On** indicating that pace pulse detection is On.

ECG Sweep Speed

The ECG Sweep Speed parameter allows you to set the ECG sweep speed.

The values for the ECG Sweep Speed parameter are:

Possible Values	25 mm/s, 50 mm/s
Default	25 mm/s

NIBP Monitoring Parameters

NIBP monitoring parameters allow you to view and modify the NIBP settings of the X Series unit. These parameters include NIBP Cuff Inflation Target.

NIBP Cuff Inflation Target

The NIBP Cuff Inflation Target parameter allows you to select the default cuff inflation target pressure when you select NIBP monitoring.

The values for the NIBP Cuff Inflation Target parameter are:

Possible Values	Adult: 120, 140, 160, 180, 200, 220, 240, 260 Pediatric: 80, 90, 100, 110, 120, 130, 140, 150 Neonate: 60, 70, 80, 90, 100, 110, 120, 130
Default	Adult: 160 Pediatric: 120 Neonate: 90

Display/Volume Parameters

The Display/Volume parameters allow you to view and modify the visual and audio settings for the unit. These parameters include:

- Display Brightness
- HR/PR Tone
- Volume

Display Brightness

The Display Brightness parameter allows you to set the brightness level for the monitor display.

The values for the Display Brightness parameter are:

Possible Values	10%, 20%, 30%, 40%, 50%, 60%, 70%, 80%, 90%, 100%
Default	70%

HR/PR Tone

The HR/PR Tone parameter allows you to enable or disable the heart rate and pulse rate tone on the unit.

The values for the HR/PR Tone parameter are:

Possible Values	On, Off
Default	On

Volume

The Volume parameter allows you to set the volume level for the X Series unit and audio prompts.

The values for the Volume parameter are:

Possible Values	Low, Medium, High
Default	Medium

 **WARNING!** Do not adjust the alarm signal volume lower than the ambient noise level as this may impede operator recognition of alarm signals.

Printer Parameters

The Printer parameters allow you to view and modify settings related to the printing of recorded data. These parameters include:

- Print Speed
- Print Number of Traces
- Print Grid
- Print on Snapshot
- Print on Patient Alarm
- Print on NIBP
- Print on Trend

Print Speed

The Print Speed parameter allows you to select how fast data is printed.

The values for the Print Speed parameter are:

Possible Values	25 mm/s, 50 mm/s
Default	25 mm/s

Print Number of Traces

The Print Number of Traces parameter allows you to set how many traces are printed.

The values for the Print Number of Traces parameter are:

Possible Values	1, 2, 3, 4
Default	3

Print Grid

The Print Grid parameter allows you to specify that the data is printed on a grid.

The values for the Print Grid parameter are:

Possible Values	On, Off
Default	On

Print on Snapshot

The Print on Snapshot parameter allows you to enable or disable the printing of snapshots.

The values for the Print on Snapshot parameter are:

Possible Values	On, Off
Default	Off

Print on Patient Alarm

The Print on Patient Alarm parameter allows you to determine whether or not to print each time an alarm occurs.

The values for the Print on Patient Alarm parameter are:

Possible Values	On, Off
Default	Off

Print on NIBP

The Print on NIBP parameter allows you to determine whether or not to print NIBP measurements each time the unit takes them.

The values for the Print on NIBP parameter are:

Possible Values	On, Off
Default	Off

Print on Trend

The Print on Trend parameter allows you to determine whether or not to print trend values each time the unit takes a trend data snapshot.

The values for the Print on Trend parameter are:

Possible Values	On, Off
Default	Off

Trends Parameters

The Trends parameters allow you to view and modify settings related to trend information and all monitored vital sign measurements. These parameters include:

- Trend Display Format
- Trend Display Interval
- Trend on Alarm

Trend Display Format

The Trend Display Format parameter allows you to specify what vital sign measurements appear on the Trends status monitor.

The values for the Trend Display Format parameter are:

Possible Values	Resp, SpO ₂ , Temp, P1, P2, P3, NIBP
Default	NIBP

- **Resp** Displays the following vital signs: HR, SpO₂, RR, EtCO₂, FiCO₂
- **SpO₂** Displays the following vital signs: HR, SpO₂, SpCO[®], SpMet[®]
- **SpHb** Displays the following vital signs: HR, SpO₂, SpHb, SpOC, PVI
- **Temp** Displays the following vital signs: HR, SpO₂, T1, T2, ΔT
- **P1** Displays the following vital signs: HR, SpO₂, IBP1, RR
- **P2** Displays the following vital signs: HR, SpO₂, IBP2, RR
- **P3** Displays the following vital signs: HR, SpO₂, IBP3, RR
- **NIBP** Displays the following vital signs: HR, SpO₂, NIBP, RR

Trend Display Interval

The Trend Display Interval parameter allows you to specify the time interval to display trends.

The values for the Trend Display Interval parameter are:

Possible Values	30 sec, 1 min, 5 min, 10 min, 15 min, 30 min, 60 min
Default	5 min

Trend on Alarm

The Trend on Alarm parameter allows you to specify whether or not to log Trends each time an alarm occurs.

The values for the Trend on Alarm parameter are:

Possible Values	On, Off
Default	Off

Operational Tests

The Operational Tests allow you to ensure that the X Series unit is operating at optimal efficiency, and include:

- Keypad Test
- LED Test
- LCD Test
- RFU Indicator Test
- Audio Test
- Printer Test

Device Info


The Device Info screen displays specific technical information about the X Series unit, such as:

- Application Version
- Serial Number
- Unit ID
- Department Name
- Contact Telephone

Readiness Test Log Review

In addition to displaying technical information, the Device Info screen allows you to print or transmit the Readiness Test logs to a Defib Dashboard server. The X Series unit performs a readiness test when the unit is turned on. The test verifies that the unit is ready in the event of a code.

To transmit or print a Readiness Test log:

1. Access the Setup menu by pressing the **Setup** () quick access key then use the navigation keys to highlight and select **Device Info**. The Device Info screen displays.
2. Select **Readiness Test Logs Review** button. The Readiness Test Logs Review screen displays with a list of up to 20 of the most recent Readiness Logs.
3. Select a Readiness Test log to transmit or print by navigating to and selecting a log from the list. A green check box indicates a selected log.



Note: You can only select one log at a time to print or transmit.

4. If you would like to transmit the log to the configured server, select the **Transmit** button. The X Series unit begins transmission of the selected log. See "Device Info" above
5. If you would like to print the log, select the **Print** button. The X Series unit begins printing the currently selected log.

CHAPTER 3

Parameter Menus


This chapter describes the configuration settings you can modify in the Parameter Menus.

It includes the following sections:

Accessing the Parameter Menus	24
HR/PR Parameter Settings	25
NIBP Parameter Settings	28
Resp Parameter Settings	34
SpO2 Parameter Settings	38
Temp Parameter Settings	51
IBP1 Parameter Settings	55
IBP2 Parameter Settings	60
IBP3 Parameter Settings	65

Accessing the Parameter Menus

All users can access the Parameter menus, which allow you to modify the monitoring protocols of the unit. The Parameter menus share many of the same settings as the Supervisor menu. If the shared settings have been configured in the Supervisor menu, they can be modified in the Parameter menus, but they will default back to the Supervisor settings once the unit is turned off for two minutes.

You can access these menus in the monitoring window by pressing the **Display/Home** button () and then use the navigation keys to highlight and select the appropriate settings in the menus.

The Parameter menus include the following parameters:

- "HR/PR Parameter Settings" on the facing page
- "NIBP Parameter Settings" on page 28
- "Resp Parameter Settings" on page 34
- "SpO2 Parameter Settings" on page 38
- "Temp Parameter Settings" on page 51
- "IBP1 Parameter Settings" on page 55
- "IBP2 Parameter Settings" on page 60
- "IBP3 Parameter Settings" on page 65

HR/PR Parameter Settings

The HR/PR parameter settings allow you to view and modify functions related to using the unit to monitor heart rate. These parameters include:

- HR/PR
- Extreme Brady/Tachy Alarms
- LTA (Life Threatening Alarms) Monitoring
- HR/PR Tone
- HR/PR Selected Source
- Current Source
- ECG Sweep Speed

HR/PR Alarm

The following options allow you to set the limits of the heart rate/pulse rate alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the HR/PR alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the HR/PR alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the HR/PR alarm.

The values for the Upper Limit parameter are:

Possible Values	32-300
Default	Adult: 120 Pediatric: 150 Neonate: 200

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the HR/PR alarm.

The values for the Lower Limit parameter are:

Possible Values	30-298
Default	Adult: 50 Pediatric: 50 Neonate: 100

Extreme Brady/Tachy Alarms

If LTA monitoring is enabled, the Extreme Brady/Tachy Alarms parameter allows you to set the limits of the Extreme Brady/Tachy alarms.

The values for the Extreme Brady/Tachy Alarm parameter are:

Possible Values	Tachy: +5-+30 by 5's Brady: -5--30 by 5's
Default	Tachy: +20 Brady: -10

LTA Monitoring

The LTA Monitoring parameter allows you to enable or disable LTA monitoring.

The values for the LTA Monitoring parameter are:

Possible Values	On, Off
Default	Off

HR/PR Tone

The HR/PR Tone parameter allows you to enable or disable the HR/PR tone.

The values for the HR/PR Tone parameter are:

Possible Values	On, Off
Default	On

HR/PR Selected Source

The HR/PR Selected Source parameter allows you to specify the HR/PR source.

The values for the HR/PR Selected Source parameter are:

Possible Values	ECG, SpO2, P1, P2, P3
Default	ECG

ECG Sweep Speed

The ECG Sweep Speed parameter allows you to set the ECG sweep speed.

The values for the ECG Sweep Speed parameter are:

Possible Values	25 mm/s, 50 mm/s
Default	25 mm/s

NIBP Parameter Settings

The NIBP parameter settings allow you to view and modify functions related to using the unit for NIBP monitoring, and includes:

- NIBP Systolic Alarm
- NIBP Diastolic Alarm
- NIBP MAP Alarm
- NIBP Mode
- NIBP Auto Mode Interval
- SmartCuf On/Off

NIBP Systolic Alarm

The following options allow you to set the limits of the NIBP Systolic Alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the NIBP systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	Adult: 32-260 Pediatric: 32-160 Neonate: 22-120
Default	Adult: 220 Pediatric: 145 Neonate: 100

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	Adult: 30-258 Pediatric: 30-158 Neonate: 20-118
Default	Adult: 75 Pediatric: 75 Neonate: 50

NIBP Diastolic Alarm

The following options allow you to set the limits of the NIBP Diastolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the NIBP diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	Adult: 22-220 Pediatric: 22-130 Neonate: 12-110
Default	Adult: 110 Pediatric: 100 Neonate:70

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	Adult: 20-218 Pediatric: 20-128
------------------------	------------------------------------

	Neonate: 10-108
Default	Adult: 35 Pediatric: 35 Neonate: 30

NIBP MAP Alarm

The following options allow you to set the limits of the NIBP MAP alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the NIBP MAP alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP MAP alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP MAP alarm.

The values for the Upper Limit parameter are:

Possible Values	Adult: 25-23 Pediatric: 25-140 Neonate: 15-110
Default	Adult: 120 Pediatric: 110 Neonate: 80

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP MAP alarm.

The values for the Lower Limit parameter are:


Possible Values	Adult: 23-228 Pediatric: 23-138 Neonate: 13-108
Default	Adult: 50 Pediatric: 35 Neonate: 35

NIBP Mode

The NIBP Mode parameter allows you to set the NIBP Mode.

The values for the NIBP Mode parameter are:

Possible Values	Manual, Auto
Default	Manual

- **Manual:** Unit takes a single NIBP measurement when you press the **NIBP** button  on the front panel. To repeat, press the **NIBP** key again.
- **Auto:** Unit takes the first of a series of NIBP measurements when the **NIBP Auto Mode Interval** timer expires, and then repeats the NIBP measurement at this specified interval.

NIBP Auto Mode Interval

If Auto Mode is selected, the NIBP Auto Mode Interval parameter allows you to set the interval time.

The values for the NIBP Auto Mode Interval parameter are:

Possible Values	1 min, 2 min, 3 min, 5 min, 10 min, 15 min, 30 min, 60 min
Default	5 min

SmartCuf On/Off

The SmartCuf On/Off parameter allows you to enable or disable SmartCuf.

The values for the SmartCuf On/Off parameter are:

Possible Values	On, Off
Default	On

Resp Parameter Settings

The Resp parameter settings allow you to view and modify functions related to using the X Series unit for respiration monitoring, and include:

- RR/BR Alarm
- No Breath Alarm
- Resp Monitoring
- Resp Lead
- CO2/Resp Sweep Speed

RR/BR Alarm

The following options allow you to set the limits of the Respiration Rate alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the RR_BR alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the RR_BR alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the RR_BR alarm.

The values for the Upper Limit are:

Possible Values	5-150
Default	Adult: 50 Pediatric: 50 Neonate: 80

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the RR_BR alarm.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	Adult: 3 Pediatric: 3 Neonate: 12

No Breath Alarm

The No Breath Alarm parameter allows you to set the time of the No Breath alarm for all patient age groups.

The values for the No Breath Alarm parameter are:

Possible Values	Adult: Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec Pediatric: Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec Neonate: Off, 10 sec, 15 sec, 20 sec
Default	Adult: Off Pediatric: Off Neonate: Off

Resp Monitoring

The Resp Monitoring parameter allows you to enable or disable Resp monitoring.

The values for the Resp Monitoring parameter are:

Possible Values	On, Off
Default	On

Resp Lead

The Resp Lead parameter allows you to specify the Lead cable used for Resp monitoring.

The values for the Resp Lead parameter are:

Possible Values	Lead I (RA-LA), Lead II (RA-LL)
Default	Lead II (RA-LL)

CO₂/Resp Sweep Speed

The CO₂/Resp Sweep Speed parameter allows you to set the CO₂/Resp sweep speed.

The values for the CO₂/Resp Sweep Speed parameter are:

Possible Values	3.13 mm/s, 6.25 mm/s, 12.5 mm/s
Default	12.5 mm/s

SpO2 Parameter Settings

The SpO2 parameter settings allow you to view and modify functions related to Pulse CO-Oximetry monitoring, and include:

- SpO2 Alarm
- SpCO Alarm
- SpMet Alarm
- SpHb Alarm
- SpOC Alarm
- PVI Alarm
- PI Alarm
- SpCO Monitoring
- SpMet Monitoring
- SpHb Monitoring
- SpOC Monitoring
- PVI Monitoring
- PI Monitoring
- Averaging Time
- Sensitivity
- HR/PR Tone

SpO2 Alarm

The following options allow you to set the limits of the SpO2 alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpO2 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpO2 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpO2 alarm.

The values for the Upper Limit parameter are:

Possible Values	52-100
Default	Adult: 100 Pediatric: 100 Neonate: 95

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpO2 alarm.

The values for the Lower Limit parameter are:

Possible Values	50-98
Default	85

SpCO Alarm

The following options allow you to set the limits of the SpCO alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpCO alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	On

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpCO alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpCO alarm.

The values for the Upper Limit parameter are:

Possible Values	2-100
Default	Adult: 10 Pediatric: 10 Neonate: 10

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpCO alarm.

The values for the Lower Limit parameter are:

Possible Values	0-98
Default	Adult: 0 Pediatric: 0 Neonate: 0

SpMet Alarm

The following options allow you to set the limits of the SpMet alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpMet alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	On

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpMet alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpMet alarm.

The values for the Upper Limit parameter are:

Possible Values	2.0, 2.5, 3.0, 3.5, 4.0, 4.5, 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5, 10.0-100.0
Default	Adult: 3.0 Pediatric: 3.0 Neonate: 3.0

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpMet alarm.

The values for the Lower Limit parameter are:

Possible Values	0.0, 0.5, 1.0, 1.5, 2.0, 2.5, 3.0, 3.5, 4.0, 4.5, 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5, 10.0-98.0
Default	Adult: 0.0 Pediatric: 0.0 Neonate: 0.0

SpHb Alarm

The following options allow you to set the limits of the SpHb alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpHb alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpHb alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpHb alarm.

The values for the Upper Limit parameter are:

Possible Values	gdL: 2 to 25 mmol/L: 2 to 15.5
Default	Adult: 17.0 gdL 11.0 mmol/L Pediatric: 17.0 gdL 11.0 mmol/L Neonate: 17.0 gdL 11.0 mmol/L

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpHb alarm.

The values for the Lower Limit parameter are:

Possible Values	gdL: 0 to 24.9 mmol/L: 0 to 15.4
Default	Adult: 7.0 gdL 4.0 mmol/L Pediatric: 7.0 gdL 4.0 mmol/L Neonate: 7.0 gdL 4.0 mmol/L

SpOC Alarm

The following options allow you to set the limits of the SpOC alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpOC alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpOC alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpOC alarm.

The values for the Upper Limit parameter are:

Possible Values	0.1 to 35 ml/dL
Default	Adult: 25 ml/dL Pediatric: 25 ml/dL Neonate: 25 ml/dL

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpOC alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 34.9 ml/dL
Default	Adult: 10 Pediatric: 10 Neonate: 10

PVI Alarm

The following options allow you to set the limits of the PVI alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the PVI alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the PVI alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the PVI alarm.

The values for the Upper Limit parameter are:

Possible Values	2 to 100%
Default	40%

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the PVI alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 98%
Default	5%

PI Alarm

The following options allow you to set the limits of the PI alarm (if enabled) for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the PI alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the PI alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the PI alarm.

The values for the Upper Limit parameter are:

Possible Values	0.2 to 20%
Default	Adult: 20% Pediatric: 20% Neonate: 20%

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the PI alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 19.8%
Default	Adult: 0% Pediatric: 0% Neonate: 0%

SpCO Monitoring

The SpCO Monitoring parameter allows you to enable or disable SpCO monitoring. SpCO monitoring is not available on units with the SpO2 Nellcor module installed.

The values for the SpCO Monitoring parameter are:

Possible Values	On, Off
Default	Off

SpMet Monitoring

The SpMet Monitoring parameter allows you to enable or disable SpMet monitoring. SpMet monitoring is not available on units with the SpO2 Nellcor module installed.

The values for the SpMet Monitoring parameter are:

Possible Values	On, Off
Default	Off

SpHb Monitoring


The SpHb Monitoring parameter allows you to enable or disable SpHb monitoring. SpHb monitoring is not available on units with the SpO2 Nellcor module installed.

The values for the SpHb Monitoring parameter are:

Possible Values	On, Off
Default	Off

SpOC Monitoring

The SpOC Monitoring parameter allows you to enable or disable SpOC monitoring.

 **Note:** SpOC monitoring is only available when SpHb is installed.

The values for the SpOC Monitoring parameter are:

Possible Values	On, Off
Default	Off

PVI Monitoring

The PVI Monitoring parameter allows you to enable or disable PVI monitoring.

The values for the PVI Monitoring parameter are:

Possible Values	On, Off
Default	Off

PI Monitoring

The PI Monitoring parameter allows you to enable or disable PI monitoring.

The values for the PI Monitoring parameter are:

Possible Values	On, Off
Default	Off

Averaging Time

The Averaging Time parameter allows you to specify the SpO2 averaging time.

The values for the Averaging Time parameter are:

Possible Values	4 seconds, 8 seconds, 16 seconds
Default	8 seconds

Sensitivity


The Sensitivity parameter allows you to specify the sensitivity level of the SpO2 monitoring.

The values for the Sensitivity parameter are:

Possible Values	Normal, Maximum
Default	Normal

SpHb Venous Mode

The SpHb Venous Mode parameter allows you to specify whether to use venous mode as the blood sample source for SpHb measurements.

 **Note:** The X Series unit uses arterial mode when the SpHb Venous Mode parameter is set to **Off**.

The values for the Venous Mode parameter are:

Possible Values	On, Off
Default	Off

SpHb Averaging Time

The SpHb Averaging Time parameter allows you to choose the level of visibility to variations in SpHb values.

The values for the SpHb Averaging Time parameter are:

Possible Values	Short, Medium, Long
Default	Long

HR/PR Tone

The HR/PR parameter allows you to enable or disable heart rate/pulse rate monitoring.

The values for the HR/PR Tone parameter are:

Possible Values	On, Off
Default	On

Temp Parameter Settings

The Temp parameters allow you to view and modify functions related to using the X Series unit for temperature monitoring. These parameters include:

- T1 Alarm
- T2 Alarm
- ΔT Alarm
- T1 Source Label
- T2 Source Label

T1 Alarm

The following options allow you to set the limits of the T1 alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the T1 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the T1 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the T1 alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the T1 alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

T2 Alarm

The following options allow you to set the limits of the T2 alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the T2 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the T2 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the T2 alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the T2 alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

 ΔT Alarm

The following options allow you to set the limits of the ΔT Alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the ΔT Alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the ΔT Alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the ΔT Alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 0.2-90.0 °C: 02-50.0
Default	°F: 5.0 °C: 2.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the ΔT Alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 0.0-89.8 °C: 0.0-49.8
Default	°F: 0.0 °C: 0.0

T1 Source Label

The T1 Source Label parameter allows you to specify the label for the T1 monitor.

The values for the T1 Source Label parameter are:

Possible Values	T1, ART, CORE, ESOPH, RECT, SKIN, VEN, NASO
Default	T1

T2 Source Label

The T2 Source Label parameter allows you to specify the label for the T2 monitor.

The values for the T2 Source Label parameter are:

Possible Values	T2, ART, CORE, ESOPH, RECT, SKIN, VEN, NASO
Default	T2

IBP1 Parameter Settings

The IBP1 parameter settings allow you to view and modify functions related to the first channel of the X Series unit for invasive blood pressure monitoring, and include:

- P1 Systolic Alarm
- P1 Diastolic Alarm
- P1 Mean Alarm
- Source Label
- Display Format

P1 Systolic Alarm

The following options allow you to set the limits of the P1 Systolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P1 systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P1 systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P1 systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 220 Pediatric: 145 Neonate: 100

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P1 systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 75 Pediatric: 75 Neonate: 50

P1 Diastolic Alarm

The following options allow you to set the limits of the P1 Diastolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit of the NIBP diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit of the NIBP diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit of the NIBP diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 110 Pediatric:100 Neonate: 70

Lower Limit

The Lower Limit parameter allows you to set the lower limit of the NIBP diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 35 Pediatric: 35 Neonate: 30

P1 Mean Alarm

The following options allow you to set the limits of the P1 Mean alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 120 Pediatric: 110 Neonate: 80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 50 Pediatric: 50 Neonate: 35

Source Label

The Source Label parameter allows you to specify the label of the P1 channel.

The values for the Source Label parameter are:

Possible Values	P1, ABP, AO, ART, BAP, CVP, FAP, ICP, LAP, PAP, RAP, UAP, UVP
Default	P1

Display Format

The Display Format parameter allows you to specify whether to display MAP measurement (M) with or without the Systolic (S) and Diastolic (D) measurements.

The values for the Display Format parameter are:

Possible Values	S/D (M), S/D, (M) S/D, (M)
Default	S/D (M)

IBP2 Parameter Settings

The IBP2 parameter settings allow you to view and modify functions related to the second channel of the X Series unit for invasive blood pressure monitoring, and include:

- P2 Systolic Alarm
- P2 Diastolic Alarm
- P2 Mean Alarm
- Source Label
- Display Format

P2 Systolic Alarm

The following options allow you to set the limits of the P2 Systolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P2 systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P2 systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P2 systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 220 Pediatric: 145 Neonate: 100

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P2 systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 75 Pediatric: 75 Neonate: 50

P2 Diastolic Alarm

The following options allow you to set the limits of the P2 Diastolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P2 diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P2 diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Of

Upper Limit

The Upper Limit parameter allows you to set the upper limit of the for the P2 diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 110 Pediatric: 100 Neonate: 70

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P2 diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 35 Pediatric: 35 Neonate: 30

P2 Mean Alarm

The following options allow you to set the limits of the P2 Mean alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 120 Pediatric: 110 Neonate: 80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 50 Pediatric: 50 Neonate: 35

Source Label

The Source Label parameter allows you to specify the label of the P2 channel.

The values for the Source Label parameter are:

Possible Values	P2, ABP, AO, BAP, CVP, FAP, ICP, LAP, PAP, RAP, UAP, UVP
Default	P2

Display Format

The Display Format parameter allows you to specify whether to display MAP measurement (M) with or without the Systolic (S) and Diastolic (D) measurements.

The values for the Display Format parameter are:

Possible Values	S/D (M), S/D, (M) S/D, (M)
Default	S/D (M)

IBP3 Parameter Settings

The IBP3 parameter settings allow you to view and modify functions related to the third channel of the X Series unit for invasive blood pressure monitoring, and include:

- P3 Systolic Alarm
- P3 Diastolic Alarm
- P3 Mean Alarm
- Source Label
- Display Format

P3 Systolic Alarm

The following options allow you to set the limits of the P3 Systolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P3 systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 220 Pediatric: 145 Neonate: 100

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 75 Pediatric: 75 Neonate: 50

P3 Diastolic Alarm

The following options allow you to set the limits of the P3 Diastolic alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P3 diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 110 Pediatric: 100 Neonate: 70

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 35 Pediatric: 35 Neonate: 30

P3 Mean Alarm

The following options allow you to set the limits of the P3 Mean alarm for all patient age groups.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 Mean alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P3 Mean alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 Mean alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	Adult: 120 Pediatric: 110 Neonate: 80

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 Mean alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	Adult: 50 Pediatric: 50 Neonate: 35

Source Label

The Source Label parameter allows you to specify the label of the third channel.

The values for the Source Label parameter are:

Possible Values	P3, ABP, AO, ART, BAP, CVP, FAP, ICP, PAP, RAP, UAP, UVP
Default	P3

Display Format

The Display Format parameter allows you to specify whether to display MAP measurement (M) with or without the Systolic (S) and Diastolic (D) measurements.

The values for the Display Format parameter are:

Possible Values	S/D (M), S/D, (M) S/D, (M)
Default	S/D (M)

CHAPTER 4

Supervisor Menu



This chapter describes the configuration settings you can modify in the Supervisor menu.

It includes the following sections:

Accessing the Supervisor Menu	71
Alarms Parameters	72
Log Parameters	140
Defib/Pacer Parameters	144
ECG Parameters	149
NIBP Parameters	153
Printer Parameters	155
Display/Configuration Parameters	158
AED/Advisory Parameters	173
CPR Parameters	182
Communications Parameters	188
Service Parameters	210

Accessing the Supervisor Menu

The Supervisor menu allows you to configure default settings such as alarms, NIBP, and Defib/Pacer. Access to the Supervisor menu is passcode-protected and only users authorized to view and modify the settings may do so.

To access the Supervisor menu, press the **More** () and **Setup** () quick access keys, and select **Supervisor**. You are then required to enter the four digit Supervisor passcode before accessing the configurable options in the menus.

The Supervisor menu includes the following parameter settings.

- "Alarms Parameters" on the facing page
- "Log Parameters" on page 140
- "Defib/Pacer Parameters" on page 144
- "ECG Parameters" on page 149
- "NIBP Parameters" on page 153
- "Printer Parameters" on page 155
- "Display/Configuration Parameters" on page 158
- "AED/Advisory Parameters" on page 173
- "CPR Parameters" on page 182
- "Communications Parameters" on page 188
- "Clinical Dashboards" on page 1
- "Service Parameters" on page 210

Alarms Parameters

The Alarm parameters allow you to view and modify functions related with Alarm states and thresholds for the X Series unit. The main level parameters in this menu include:

- General
- Default Adult
- Default Pediatric
- Default Neonate

General

The following options allow you to specify basic conditions for all alarms.

Volume

The Volume parameter allows you to set the volume level for all alarms.

The values for the Volume parameter are:

Possible Values	Low, Medium, High
Default	Medium

Audio Paused Time (Alarm Suspend Time)

The Audio Paused Time parameter allows you to specify how long any alarm is paused (suspended). No alarms sound during the pause time period, however, if an alarm occurs during this period, the visual alarm indicators display.

The values for the Audio Paused Time parameter are:

Possible Values	Disable, 2 min, 4 min, 15 min
Default	4 min

Alarm Latching Enable

The Alarm Latching Enable parameter allows you to turn alarm latching on or off for high priority patient alarms. When alarms are configured to be latching, alarm indicators (sound, message, color) remain, whether or not the condition is present. Latched alarms must be acknowledged even if the condition no longer exists. When alarms are configured as non-latching alarm indicators clear when the alarm condition ends whether or not the alarm was acknowledged.

The values for Alarm Latching Enable parameter are:

Possible Values	Yes, No
Default	No

Reminder Alarm

The Alarm Reminder parameter allows you to specify whether to have a alarm reminder sound when an Audio Off condition persists. If you select **Enable**, a reminder alarm will sound every 5, 10 or 15 minutes (depending on configuration) while the Audio Off condition continues.

Possible Values	Enable, Disable
Default	Disable

Reminder Alarm Interval

The Reminder Alarm Interval parameter allows you to specify how often an alarm reminder will sound when an Audio Off condition exists.

Possible Values	5 min, 10 min, 15 min
Default	5 min

Default Adult

The following options allow you specify the alarm limits for adult patients.

LTA (Life Threatening Alarms) Monitoring

The LTA Monitoring parameter allows you to enable or disable adult LTA monitoring.

The values for the LTA Monitoring parameter are:

Possible Values	On, Off
Default	Off

HR/PR Alarm

The following options allow you to set the limits of the adult HR/PR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the HR/PR alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the HR/PR alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the HR/PR alarm.

The values for the Upper Limit parameter are:

Possible Values	32-300
Default	120

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the HR/PR alarm.

The values for the Lower Limit parameter are:

Possible Values	30-298
Default	50

NIBP Systolic Alarm

The following options allow you to set the limits of the adult NIBP Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the NIBP Systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP Systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP Systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	32-260
Default	220

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP Systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	30-258
Default	75

NIBP Diastolic Alarm

The following options allow you to set the limits of the adult NIBP Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP Diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP Diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	22-220
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP Diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	20-218
Default	35

NIBP MAP Alarm

The following options allow you to set the limits of the adult NIBP MAP alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the NIBP MAP alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the NIBP MAP alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the NIBP MAP alarm..

The values for the Upper Limit parameter are:

Possible Values	25-230
Default	120

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the NIBP MAP alarm.

The values for the Lower Limit parameter are:

Possible Values	23-228
------------------------	--------

Default	50
----------------	----

EtCO₂ Alarm

The following options allow you to set the limits of the adult EtCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the adult EtCO₂ alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the adult EtCO₂ alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the adult EtCO₂ alarm.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	60

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the adult EtCO₂ alarm.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	8

FiCO₂ Alarm

The following options allow you to set the limits of the adult FiCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the FiCO2 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the FiCO2 alarm.

The values for the Upper Limit parameter are:

Possible Values	2-98
Default	8

RR/BR Alarm

The following options allow you to set the limits of the adult RR/BR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the RR/BR alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit t for the RR/BR alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the RR/BR alarm.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	50

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the RR/BR alarm.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	3

No Breath Alarm

The No Breathe Alarm parameter allows you to specify the time of the adult No Breath alarm.

The values for the No Breath Alarm are:

Possible Values	Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec
Default	Off

SpO2 Alarm

The following options allow you to set the limits of the adult SpO2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpO2 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpO2 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
------------------------	---------

Default	Off
----------------	-----

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpO2 alarm.

The values for the Upper Limit parameter are:

Possible Values	52-100
Default	100

Lower Limit


The Lower Limit parameter allows you to set the lower limit for the SpO2 alarm.

The values for the Lower Limit parameter are:

Possible Values	50-98
Default	85

SpCO Alarm

The following options allow you to set the limits of the adult SpCO alarm.

 **Note:** The SpCO Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpCO alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpCO alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2-100
Default	10

Lower Limit


The Lower Limit parameter allows you to set the lower limit for the SpCO alarm.

The values for the Lower Limit parameter are:

Possible Values	1-98
Default	0

SpMet Alarm

The following options allow you to set the limits of the adult SpMet alarm.

 **Note:** The SpMet Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpMet alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpMet alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpMet alarm.

The values for the Upper Limit parameter are:

Possible Values	2.0-100.0
------------------------	-----------

Default	3.0
----------------	-----

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0.0-98.0
Default	0.0

SpHb Alarm

The following options allow you to set the limits of the adult SpHb alarm.

 **Note:** The SpHb Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the SpHb alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpHb alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpHb alarm.

The values for the Upper Limit parameter are:

Possible Values	g/dL: 2 to 24.5 mmol/L: 2 to 15
Default	17.0 g/dL 11.0 mmol/L

Lower Limit


The Lower Limit parameter allows you to set the lower limit for the SpHb alarm.

The values for the Lower Limit parameter are:

Possible Values	g/dL: 0 to 24.9 mmol/L: 0 to 15.4
Default	7.0 g/dL 4.0 mmol/L

SpOC Alarm

The following options allow you to set the limits of the adult SpOC alarm.

 **Note:** The SpOC Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the SpOC alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the SpOC alarm.

The values for the Upper Limit parameter are:

Possible Values	0.1 to 35 ml/dL
Default	25 ml/dL

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the SpOC alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 34.9 ml/dL
Default	10 ml/dL

PVI Alarm

The following options allow you to set the limits of the adult PVI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the PVI alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the PVI alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the PVI alarm.

The values for the Upper Limit parameter are:

Possible Values	2 to 100%
Default	40%

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the PVI alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 98%
Default	5%

PI Alarm

The following options allow you to set the limits of the adult PI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the PI alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the PI alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the PI alarm.

The values for the Upper Limit parameter are:

Possible Values	0.2 to 20%
Default	20%

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the PI alarm.

The values for the Lower Limit parameter are:

Possible Values	0 to 19.8%
Default	0%

T1 Alarm

The following options allow you to set the limits of the adult T1 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the T1 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the T1 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the T1 alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the T1 alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

T2 Alarm

The following options allow you to set the limits of the adult T2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the T2 alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
------------------------	---------

Default	Off
----------------	-----

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the T2 alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the T2 alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the T2 alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

 ΔT Alarm

The following options allow you to set the limits of the adult ΔT alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the ΔT alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the ΔT alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the ΔT alarm.

The values for the Upper Limit parameter are:

Possible Values	°F: 0.2-90.0 °C: 0.2-50.0
Default	°F: 5.0 °C: 2.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the ΔT alarm.

The values for the Lower Limit parameter are:

Possible Values	°F: 0.0-89.8 °C: 0.0-49.8
Default	°F: 0.0 °C: 0.0

P1 Systolic Alarm

The following options allow you to set the limits of the adult P1 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P1 Systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P1 Systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P1 Systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	220

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P1 Systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P1 Diastolic Alarm

The following options allow you to set the limits of the adult P1 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P1 Diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P1 Diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P1 Diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P1 Diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P1 Mean Alarm

The following options allow you to set the limits of the adult P1 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P1 Mean alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P1 Mean alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P1 Mean alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	120

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P1 Mean alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P2 Systolic Alarm

The following options allow you to set the limits of the adult P2 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P2 Systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P2 Systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P2 Systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	220

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P2 Systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P2 Diastolic Alarm

The following options allow you to set the limits of the adult P2 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P2 Diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P2 Diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P2 Diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P2 Diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
------------------------	---------

Default	35
----------------	----

P2 Mean Alarm

The following options allow you to set the limits of the adult P2 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P2 Mean alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P2 Mean alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P2 Mean alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	120

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P2 Mean alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P3 Systolic Alarm

The following options allow you to set the limits of the adult P3 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 Systolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the upper limit for the P3 Systolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 Systolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	220

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 Systolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P3 Diastolic Alarm

The following options allow you to set the limits of the adult P3 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 Diastolic alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P3 Diastolic alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 Diastolic alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 Diastolic alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P3 Mean Alarm

The following options allow you to set the limits of the adult P3 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit for the P3 Mean alarm.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit for the P3 Mean alarm.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit for the P3 Mean alarm.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	120

Lower Limit

The Lower Limit parameter allows you to set the lower limit for the P3 Mean alarm.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

Default Pediatric

The following options allow you to specify the alarm limits for pediatric patients.

LTA Monitoring

The LTA Monitoring parameter allows you to enable or disable pediatric LTA Monitoring.

The values for the LTA Monitoring parameter are:

Possible Values	On, Off
Default	Off

HR/PR Alarm

The following options allow you to set the limits of the pediatric HR/PR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	32-300
Default	150

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	30-298
Default	50

NIBP Systolic Alarm

The following options allow you to set the limits of the pediatric NIBP Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	32-160
Default	145

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	30-158
Default	75

NIBP Diastolic Alarm

The following options allows you to set the limits of the pediatric NIBP Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	32-130
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	30-128
Default	35

NIBP MAP Alarm

The following options allow you to set the limits of the pediatric NIBP MAP alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	25-140
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	23-138
Default	50

EtCO₂ Alarm

The following options allow you to set the limits of the pediatric EtCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	60

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	8

FiCO₂ Alarm

The following options allows you to set the limits of the pediatric FiCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2-98
Default	8

RR/BR Alarm

The following options allow you to set the limits of the pediatric RR/BR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	50

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	3

No Breath Alarm

The No Breath Alarm parameter allows you to specify the time of the pediatric No Breath alarm.

The values for the No Breath Alarm parameter are:

Possible Values	Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec
Default	Off

SpO2 Alarm

The following options allow you to set the limits of the pediatric SpO2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	52-100
Default	100

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	50-98
Default	85

SpCO Alarm

The following options allow you to set the limits of the pediatric SpCO alarm.

 **Note:** The SpCO Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2-100
Default	10

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	1-98
Default	0

SpMet Alarm

The following options allow you to set the limits of the pediatric SpMet alarm.

 **Note:** The SpMet Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2.0-100.0
Default	3.0

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0.0-98.0
Default	0.0

SpHb Alarm

The following options allow you to set the limits of the pediatric SpHb alarm.

 **Note:** The SpHb Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	g/dL: 2 to 25 mmol/L: 2 to 15.5
Default	10.0 g/dL 7.0 mmol/L

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	g/dL: 0 to 24.9 mmol/L: 0 to 15.4
Default	0 g/dL 0 mmol/L

SpOC Alarm

The following options allow you to set the limits of the pediatric SpOC alarm.

 **Note:** The SpOC Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	0.1 to 35 ml/dL
Default	20 ml/dL

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 34.9 ml/dL
Default	0

PVI Alarm

The following options allow you to set the limits of the pediatric PVI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2 to 100%
Default	40%

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 98%
Default	5%

PI Alarm

The following options allow you to set the limits of the pediatric PI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	0.2 to 20%
Default	20%

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 19.8%
Default	0%

T1 Alarm

The following options allows you to set the limits of the pediatric T1 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

T2 Alarm

The following options allow you to set the limits of the pediatric T2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
------------------------	--------------------------------

Default	°F: 95.0 °C: 35.0
----------------	----------------------

ΔT Alarm

The following options allows you to set the limits of the pediatric ΔT alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 0.2-90.0 °C: 0.2-50.0
Default	°F: 5.0 °C: 2.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 0.0-89.8 °C: 0.0-49.8
Default	°F: 0.0 °C: 0.0

P1 Systolic Alarm

The following options allows you to set the limits of the pediatric P1 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	145

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P1 Diastolic Alarm

The following options allow you to set the limits of the pediatric P1 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
------------------------	---------

Default	Off
----------------	-----

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P1 Mean Alarm

The following options allow you to set the limits of the pediatric P1 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P2 Systolic Alarm

The following options allow you to set the limits of the pediatric P2 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	145

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P2 Diastolic Alarm

The following options allow you to set the limits of the pediatric P2 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P2 Mean Alarm

The following options allow you to set the limits of the pediatric P2 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P3 Systolic Alarm

The following options allow you to set the limits of the pediatric P3 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
------------------------	---------

Default	Off
----------------	-----

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	145

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	75

P3 Diastolic Alarm

The following options allows you to set the limits of the pediatric P3 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P3 Mean Alarm

The following options allow you to set the limits of the pediatric P3 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	110

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

Default Neonate

The following options allow you to specify the alarm limits for neonate patients.

HR/PR Alarm

The following options allow you to set the limits of the neonate HR/PR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	32-300
Default	200

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	30-298
Default	100

NIBP Systolic Alarm

The following options allow you to set the limits of the neonate NIBP Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	22-120
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	20-118
Default	50

NIBP Diastolic Alarm

The following options allow you to set the limits of the neonate NIBP Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	12-110
Default	70

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	10-108
Default	30

NIBP MAP Alarm

The following options allow you to set the limits of the neonate NIBP MAP alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	15-110
Default	80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	13-108
Default	35

EtCO₂ Alarm

The following options allow you to set the limits of the neonate EtCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	60

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	8

FiCO₂ Alarm

The following options allow you to set the limits of the neonate FiCO₂ alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2-98
Default	8

RR/BR Alarm

The following options allow you to set the limits of the neonate RR/BR alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	5-150
Default	80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0-145
Default	12

No Breath Alarm

The No Breath Alarm parameter allows you to specify the time of the neonate No Breath alarm.

The values for the No Breath Alarm parameter are:

Possible Values	Off, 10 sec, 15 sec, 20 sec
Default	20 sec

SpO2 Alarm

The following options allow you to set the limits of the neonate SpO2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	52-100
Default	95

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	50-98
Default	85

SpCO Alarm

The following options allow you to set the limits of the neonate SpCO alarm.

 **Note:** The SpCO Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2-100
Default	10

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	1-98
Default	0

SpMet Alarm

The following options allow you to set the limits of the neonate SpMet alarm.

 **Note:** The SpMet Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2.0-100.0
Default	3.0

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0.0-98.0
Default	0.0

SpHb Alarm

The following options allow you to set the limits of the neonate SpHb alarm.

 **Note:** The SpHb Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	g/dL: 2 to 25 mmol/L: 2 to 15.5
Default	17.0 g/dL 11.0 mmol/L

Lower Limit


The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	g/dL: 0 to 24.9 mmol/L: 0 to 15.4
Default	7.0 g/dL 4.0 mmol/L

SpOC Alarm

The following options allow you to set the limits of the neonate SpOC alarm.

 **Note:** The SpOC Alarm parameter settings are not available on units with the SpO2 Nellcor module installed.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	0.1 to 35 ml/dL
Default	20 ml/dL

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 34.9 ml/dL
Default	0

PVI Alarm

The following options allow you to set the limits of the neonate PVI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	2 to 100%
Default	40%

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 98%
Default	5%

PI Alarm

The following options allow you to set the limits of the neonate PI alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
------------------------	---------

Default	Off
----------------	-----

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	0.2 to 20%
Default	20%

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	0 to 19.8%
Default	0%

T1 Alarm

The following options allow you to set the limits of the neonate T1 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

T2 Alarm

The following options allow you to set the limits of the neonate T2 alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 34.0-122.0 °C: 2.0-50.0
Default	°F: 100.0 °C: 37.8

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 32.0-120.0 °C: 0.0-48.0
Default	°F: 95.0 °C: 35.0

 ΔT Alarm

The following options allow you to set the limits of the neonate ΔT alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	°F: 0.2-90.0 °C: 0.2-50.0
Default	°F: 5.0

	°C: 2.8
--	---------

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	°F: 0.0-89.8 °C: 0.0-49.8
Default	°F: 0.0 °C: 0.0

P1 Systolic Alarm

The following options allows you to set the limits of the neonate P1 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P1 Diastolic Alarm

The following options allow you to set the limits of the neonate P1 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	70

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	30

P1 Mean Alarm

The following options allow you to set the limits of the neonate P1 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P2 Systolic Alarm

The following options allow you to set the limits of the neonate P2 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P2 Diastolic Alarm

The following options allow you to set the limits of the neonate P2 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	70

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	30

P2 Mean Alarm

The following options allow you to set the limits of the neonate P2 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

P3 Systolic Alarm

The following options allow you to set the limits of the neonate P3 Systolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	100

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	50

P3 Diastolic Alarm

The following options allow you to set the limits of the neonate P3 Diastolic alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	70

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	30

P3 Mean Alarm

The following options allow you to set the limits of the neonate P3 Mean alarm.

Upper Limit Enable

The Upper Limit Enable parameter allows you to enable or disable the upper limit.

The values for the Upper Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Lower Limit Enable

The Lower Limit Enable parameter allows you to enable or disable the lower limit.

The values for the Lower Limit Enable parameter are:

Possible Values	On, Off
Default	Off

Upper Limit

The Upper Limit parameter allows you to set the upper limit.

The values for the Upper Limit parameter are:

Possible Values	-28-300
Default	80

Lower Limit

The Lower Limit parameter allows you to set the lower limit.

The values for the Lower Limit parameter are:

Possible Values	-30-298
Default	35

Log Parameters

The Log parameters allow you to view and modify functions related to recording data. The main level parameters in this menu include:

- Treatment Options
- Waveform Recording

Treatment Options

The following options allow you to set values related to Treatment options.

Configuration Method

The Configuration Method parameter allows you to specify from what listing to configure Treatment buttons.

The values for the Configuration Method parameter are:

Possible Values	Select From Predefined List, Define Custom Labels
Default	Select From Predefined List

Treatment Button #1

The Treatment Button #1 parameter allows you to specify what treatment option Button #1 covers.

The values for the Treatment Button #1 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	O2

Treatment Button #2

The Treatment Button #2 parameter allows you to specify what treatment option Button #2 covers.

The values for the Treatment Button #2 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	ASA

Treatment Button #3

The Treatment Button # 3 parameter allows you to specify what treatment option Button #3 covers.

The values for the Treatment Button #3 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	Nitro

Treatment Button #4

The Treatment Button #4 parameter allows you to specify what treatment option Button #4 covers.

The values for the Treatment Button #4 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	Morph

Treatment Button #5

The Treatment Button #5 parameter allows you to specify what treatment option Button #5 covers.

The values for the Treatment Button #5 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	IV

Treatment Button #6

The Treatment Button #6 parameter allows you to specify what treatment option Button #6 covers.

The values for the Treatment Button #6 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	B Block

Treatment Button #7

The Treatment Button #7 parameter allows you to specify what treatment option Button #7 covers.

The values for the Treatment Button #7 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	Lido

Treatment Button #8

The Treatment Button #8 parameter allows you to specify what treatment option Button #8 covers.

The values for the Treatment Button #8 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	MgSO4

Treatment Button #9

The Treatment Button #9 parameter allows you to specify what treatment option Button #9 covers.

The values for the Treatment Button #9 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	Valium

Treatment Button #10

The Treatment Button #10 parameter allows you to specify what treatment option Button #10 covers.

The values for the Treatment Button #10 parameter are:

Possible Values	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
Default	Sedate

Waveform Recording

The following options allow you to set values related to waveform recording.

Continuous Waveform Recording

The Continuous Waveform Recording parameter allows you to enable or disable continuous waveform recording.

The values for the Continuous Waveform Recording parameter are:

Possible Values	Disabled, Enabled
Default	Disabled

Record Displayed Waveforms

If enabled, the Record Displayed Waveform parameter allows you to specify which displayed waveforms are recorded.

The values for the Record Displayed Waveforms parameter are:

Possible Values	All Displayed Waveforms, Top Waveform Only
Default	All Displayed Waveforms

Record Additional Waveforms

If enabled, the Record Additional Waveform parameter allows you to specify what additional waveforms are recorded.

The values for the Record Additional Waveforms parameter are:

Possible Values	CO2 Capnogram, IBP and CO2 Capnogram
Default	CO2 Capnogram

Record Pads Imp. Waveform

The Record Pads Imp. Waveform parameter allows you to enable or disable pads impedance recording.

The values for the Continuous Waveform Recording parameter are:

Possible Values	Disabled, Enabled
Default	Disabled

Defib/Pacer Parameters

The Defib/Pacer parameters allow you to view and modify functions related to performing defibrillation and pacing. The main level parameters in this menu include:

- Default Settings
- Energy Escalation

Default Settings

Adult Defib Ext. Pdl. Energy

The Adult Defib Ext. Pdl. Energy parameter allows you to set the energy level of the external paddle for adult patients.

The values for the Adult Defib Ext. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
Default	120 J

Pediatric Defib Ext. Pdl. Energy

The Pediatric Defib Ext. Pdl. Energy parameter allows you to set the energy level of the external paddle for pediatric patients.

The values for the Pediatric Defib Ext. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
Default	50 J

Neonate Defib Ext. Pdl. Energy

The Neonate Defib Ext. Pdl. Energy parameter allows you to set the energy level of the external paddle for neonate patients.

The values for the Neonate Defib Ext. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
Default	50 J

Adult Defib Int. Pdl. Energy

The Adult Defib Int. Pdl. Energy parameter allows you to set the energy level of the internal paddle for adult patients.

The values for the Adult Defib Int. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J
Default	10 J

Pediatric Defib Int. Pdl. Energy

The Pediatric Defib Int. Pdl. Energy parameter allows you to set the energy level of the internal paddle for pediatric patients.

The values for the Pediatric Defib Int. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J,
Default	10 J

Neonate Defib Int. Pdl. Energy

The Neonate Defib Int. Pdl. Energy parameter allows you to set the energy level of the internal paddle for neonate patients.

The values for the Neonate Defib Int. Pdl. Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J
Default	10 J

Sync After Cardioversion

The Sync After Cardioversion parameter allows you to enable or disable synchronizing the defibrillator discharge with the ECG R-wave.

The values for the Sync After Cardioversion parameter are:

Possible Values	On, Off
Default	Off

Default Pacer Mode

The Default Pacer Mode parameter allows you to set the default Pacer Mode.

The values for the Default Pacer Mode parameter are:

Possible Values	Demand, Fixed
Default	Demand

- **Demand:** pacing pulses are inhibited by the patients QRS complexes that occur during an interval that is dependent on the setting of the rate control. If during that interval no QRS complexes are detected, a pacing pulse is delivered to the patient, the number of which and the rate are selected in the pacing rate window.
- **Fixed:** pacing pulses are not dependent on the patient's cardiac activity and are delivered at the selected pacing rate.

Default Pacer Rate


The Default Pacer Rate parameter allows you to set the default Pacer rate.

The values for the Default Pacer Rate parameter are:

Possible Values	30-180; <100 by 5's, >100 by 10's
Default	70

Auto Display Pads/Paddles

The Auto Display Pads/Paddles parameter allows you to enable/disable the display of Pads or Paddles as the top waveform trace when the Defibrillation Control Panel is active.

 **Note:** This parameter is only applicable to units with CP version 02.08.011.4084 and MP/PP version 02.18.03.00 or higher.

The values for the Auto Display Pads/Paddles parameter are:

Possible Values	Yes, No
Default	Yes

Energy Escalation

The following options allow you to set the energy levels of the shocks given.

Basic Auto Energy Selection

The Basic Auto Energy Selection parameter automatically escalates the defibrillator energy settings in Manual mode. This feature will operate only when Pads or Paddles are connected. Connecting internal paddles at any time during Basic Auto Energy Escalation will reset the energy back to the Internal Paddle energy level selected in Default Settings.

When enabled, the Adult Defib Ext. Pdl Energy and Pediatric Defib Ext. Pdl. Energy parameters in Default Settings are grayed out.

The values for Basic Auto Energy Selection are:

Possible Values	Disabled, Enabled
Default	Disabled

Adult Shock 1 Energy

The Adult Shock 1 Energy parameter allows you to set the energy level of the first shock given for adult patients.

The values for the Adult Shock 1 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	120 J

Adult Shock 2 Energy

The Adult Shock 2 Energy parameter allows you to set the energy level of the second shock given for adult patients.

The values for the Adult Shock 2 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	150 J

Adult Shock 3 Energy

The Adult Shock 3 Energy parameter allows you to set the energy level of the third shock given for adult patients.

The values for the Adult Shock 3 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	200 J

Pediatric Shock 1 Energy

The Pediatric Shock 1 Energy parameter allows you to set the energy level of the first shock given for pediatric patients.

The values for the Pediatric Shock 1 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	50 J

Pediatric Shock 2 Energy

The Pediatric Shock 2 Energy parameter allows you to set the energy level of the second shock given for pediatric patients.

The values for the Pediatric Shock 2 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	70 J

Pediatric Shock 3 Energy

The Pediatric Shock 3 Energy parameter allows you to set the energy level of the third shock given for pediatric patients.

The values for the Pediatric Shock 3 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	85 J

ECG Parameters

The ECG parameters allow you to view and modify functions related to ECG monitoring. The main level parameters in this menu include:

- General
- 12-Lead ECG

General

The following options allow you to specify basic conditions for all alarms.

12-Lead ECG

The following options allow you to set values related to the X Series unit's 12-Lead ECG cables.

Print on 12-Lead Acquire

The Print on 12-Lead Acquire parameter allows you to enable or disable printing of data acquired by the 12-Lead electrode cables.

The values for the Print on 12-Lead Acquire parameter are:

Possible Values	On, Off
Default	Off

Print Copies

The Print Copies parameter allows you to specify how many copies of data are printed.

The values for the Print Copies parameter are:

Possible Values	1, 2, 3, 4, 5
Default	1

12-Lead Format


The 12-Lead Format parameter allows you to specify the print format of 12-Lead data.

The values for the 12-Lead Format parameter are:

Possible Values	2x6, 3x4, 3x4 Cabrera, 3x4 Hybrid, 3x4 Cabrera Hybrid
Default	3x4

Print 10s Trace

The Print 10s Trace parameters allow you to choose to have 10 seconds of up to 3 additional waveform traces print at the end of the 12-Lead report.

 **Note:** If you are using a 3 or 5 lead cable, select V1 if you expect a V lead. If you select V2-V6 with any cable other than a 12-lead cable the unit displays the message *LEAD FAULT*.

The values for the Print 10s Trace 1 parameters are:

Possible Values	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
Default	I

The values for the Print 10s Trace 2 parameter are:

Possible Values	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
Default	Off

The values for the Print 10s Trace 3 parameter are:

Possible Values	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
Default	Off

12-Ld Frequency Response

The 12-Ld Frequency Response parameter allows you to specify the 12-Lead waveform frequency response.


The values for the 12-Ld Frequency Response parameter are:

Possible Values	Diagnostic, Filtered Diagnostic
Default	Filtered Diagnostic

- **Diagnostic:** frequency ranges are 0.525-150 Hz.
- **Filtered Diagnostic:** frequency ranges are 0.525-40 Hz.

12-Ld Analysis Enabled

The 12-Ld Analysis Enabled parameter allows you to enable or disable the 12-Lead Interpretive Analysis capability.

 **Note:** This parameter is available only if the 12-Lead Interpretive Analysis option has been purchased.

The values for the 12-Ld Analysis Enabled parameter are:

Possible Values	On, Off
Default	On


Interpretive Text

The Interpretive Text parameter allows you to choose whether or not to have 12-Lead interpretive statements display on 12-Lead Interpretive Analysis reports (both on screen and on printouts). When this parameter is set to On and 12-Lead Interpretive Analysis is enabled, the ECG analysis measurements along with interpretation statements display on the 12-Lead Interpretive Analysis report. When this parameter is set to Off only the measurements display without any interpretation statements. The setting you choose here persists through all cases and device updates.

The values for the Interpretive Text parameter are:

Possible Values	On, Off
Default	On

Patient Information Prompt

The Patient Information Prompt parameter allows you to specify whether to prompt the user to enter patient demographic data (age, gender etcetera) before initiating 12-Lead Interpretive Analysis. When the Patient Information Prompt is enabled (On) and patient age and gender has not been previously provided for the current case, the Patient Information screen displays when the Acquire quick access key  is pressed to initiate 12-Lead Interpretive Analysis. The automatic display of the Patient Information screen only occurs once per case.

The values for the Patient Information Prompt parameter are:

Possible Values	On, Off
Default	No

Print 12-Ld 1mV Bracket

The Print 12-Ld 1mV Bracket parameter allows you to enable or disable the display of the 1mV calibration bracket next to ECG waveform traces on the 12 Lead Snapshot report. If enabled the 1mV calibration bracket will appear on all 12 Lead Snapshot reports (3 x 4 Cabrera, 3 x 4, and 2 x 6).

The values for the Print 12-Ld 1mV Bracket parameter are:

Possible Values	Yes, No
Default	No

NIBP Parameters

The NIBP parameters allow you to view and modify functions related to noninvasive blood pressure monitoring. The main level parameters in this menu include:


- Default NIBP Mode
- Default Auto Mode Interval
- Default Cuff Inflation Target (Adult)
- Default Cuff Inflation Target (Ped.)
- Default Cuff Inflation Target (Neo.)

Default NIBP Mode

The Default NIBP Mode parameter allows you to set the X Series unit NIBP Mode.

The values for the Default NIBP Mode parameter are:

Possible Values	Manual, Auto
Default	Manual

- **Manual:** unit takes a single NIBP measurement when you press the **NIBP** button on the front panel (). To repeat, press the **NIBP** key again.
- **Auto:** unit takes the first of a series of NIBP measurements when the **Auto Interval** timer expires, and then repeats the NIBP measurement at this specified interval.

Default Auto Mode Interval

The Default Auto Mode Interval parameter allows you to specify the NIBP measurement interval time in Auto Mode.

The values for the Default Auto Mode Interval parameter are:

Possible Values	1 min, 2 min, 3 min, 5 min, 10 min, 15 min, 30 min, 60 min
Default	5 min

Default Cuff Inflation Target (Adult)

The Default Cuff Inflation Target (Adult) parameter allows you to set the cuff inflation target for adult patients.

The values for the Default Cuff Inflation Target (Adult) parameter are:

Possible Values	120-260; by 20's
Default	160

Default Cuff Inflation Target (Ped.)

The Default Cuff Inflation Target (Pediatric) parameter allows you to set the cuff inflation target for pediatric patients.

The values for the Default Cuff Inflation Target (Ped.) parameter are:

Possible Values	80-150; by 10's
Default	120

Default Cuff Inflation Target (Neo.)

The Default Cuff Inflation Target (Neonate) parameter allows you to set the cuff inflation target for neonate patients.

The values for the Default Cuff Inflation Target (Neo.) parameter are:

Possible Values	60-130; by 10's
Default	90

Printer Parameters

The Printer parameters allow you to view and modify what and how things are printed. The main level parameters in this menu include:

- Print on Defib/Pacer Events
- Print on Event
- Print on Treatment Snapshot
- Print Treatment Summary Snapshot
- Print on Analysis Results
- Print on Presenting Rhythm

Print on Defib/Pacer Events

The Print on Defib/Pacer Events parameter allows you to enable or disable automatic printing on Defib/Pacer events.

The values for the Print on Defib/Pacer Events parameter are:

Possible Values	On, Off
Default	On

Print on Event

The Print on Event parameter allows you to enable or disable automatic printing on events.

The values for the Print on Event parameter are:

Possible Values	On, Off
Default	Off

Print on Treatment Snapshot

The Print on Treatment Snapshot parameter allows you to enable or disable automatic printing on treatment snapshots.

The values for the Print on Treatment Snapshot parameter are:

Possible Values	On, Off
Default	Off

Print Treatment Summary Snapshots

The Print Treatment Summary Snapshots parameter allows you to enable or disable automatic printing on treatment summary snapshots.

The values for the Print Treatment Summary Snapshots parameter are:

Possible Values	On, Off
Default	On

Print on Analysis Results

The Print on Analysis Results parameter allows you to enable or disable automatic printing on analysis results.

The values for the Print on Analysis Results parameter are:

Possible Values	On, Off
Default	Off

Print on Presenting Rhythm

The Print on Presenting Rhythm parameter allows you to enable or disable automatic printing on presenting rhythm.

The values for the Print on Presenting Rhythm parameter are:

Possible Values	On, Off
Default	Off

Print On Readiness Test Results

The Print On Readiness Test Results parameter allows you to enable or disable automatic printing of readiness test results.

The values for the Print On Readiness Test Results parameter are:

Possible Values	On, Off
Default	Off

Display/Configuration Parameters

The Display/Configuration parameters allow you to view and modify what and how things appear on the X Series monitor. The main level parameters in this menu include:

- Localization
- Color
- Clock
- Device Info
- Display Setup
- Supervisor Code
- Log Clear Code

 **Note:** The Import Configuration and Export Configuration features are included in this menu, but they cannot be modified. See for information on how to use these features.

Localization

The following options allow you to set language and units in accordance to local region.

Language

The Language parameter specifies which language the X Series unit uses.

The values for the Language parameter are:

Possible Values	Arabic (Voice only), Brazilian Portuguese, Simplified Chinese, Traditional Chinese, Czech, Danish, Dutch, English, Finnish, French, French Canadian, German, Italian, Japanese, Korean (Voice only), Norwegian, Polish, Romanian, Russian, Spanish, Swedish, Turkish
Default	English

Decimal Format

The Decimal parameter allows you to set what decimal is used.

The values for the Decimal parameter are:

Possible Values	. (dot) ,(comma)
Default	. (dot)

Date Format

The Date parameter allows you to set how the date is displayed.

The values for the Date parameter are:

Possible Values	MM/DD/YYYY, DD/MM/YY, YY/MM/DD, MM-DD-YYYY, DD-MM-YYYY, YYYY-MM-DD
Default	MM/DD/YYYY

Time Format

The Time Format parameter allows you to set how the time is displayed.

The values for the Time Format parameter are:

Possible Values	12 Hour, 24 Hour
Default	24 Hour

Temp Units

The Temp parameter allows you to set what temperature unit is used.

The values for the Temp Units parameter are:

Possible Values	°F, °C
Default	°F

IBP Units

The IBP Units parameter allows you to set what unit is use for IBP monitoring.

The values for the IBP Units parameter are:

Possible Values	mmHg, kPa
Default	mmHg

CO2 Units

The CO2 Units parameter allows you to set what unit is used for CO2 monitoring.

The values for the CO2 Units parameter are:

Possible Values	mmHg, kPa, %
Default	mmHg

NIBP Units

The NIBP Units parameter allows you to set what unit is used for NIBP monitoring.

The values for the NIBP Units parameter are:

Possible Values	mmHg, kPa
Default	mmHg

ECG Lead Labels

The ECG Lead Labels parameter allows you to set the labels for the ECG Lead cables.

The values for the ECG Lead Labels parameter are:

Possible Values	AAMI, IEC
Default	AAMI

NIBP Display Format

The NIBP Display Format parameter allows you to specify whether to display MAP measurement (M) with or without the Systolic (S) and Diastolic (D) measurements.

The values for the NIBP Display Format parameter are:

Possible Values	S/D (M), (M) S/D
Default	S/D (M)

Length Units

The Length Units parameter allows you to set the length units.

The values for the Length Units parameter are:

Possible Values	Inch, cm
Default	Inch

AC Mains Filter

The AC Mains Filter parameter allows you to specify how much energy is used by the AC mains filter.

The values for the AC Mains Filter parameter are:

Possible Values	Off, 50 Hz, 60 Hz
Default	60 Hz

SpHb Units

The SpHb Units parameter allows you to specify the unit to use for measuring hemoglobin.

The values for the SpHb Units parameter are:

Possible Values	g/dL, mmol/L
Default	g/dL

Color

The following options allows you to set the display colors for specific waveforms and information displayed on the X Series monitor.

ECG Color

The ECG Color parameter allows you to specify the color for the ECG indicator on the X Series monitor.

The values for the ECG Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Green

SpO₂ Color

The SpO₂ Color parameter allows you to specify the color of the SpO₂ indicator on the X Series monitor.

The values for the SpO₂ Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

SpCO Color

The SpCO Color parameter allows you to specify the color of the SpCO indicator on the X Series monitor.

The values for the SpCO Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

SpMet Color

The SpMet Color parameter allows you to specify the color of the SpMet indicator on the X Series monitor.

The values for the SpMet Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

SpHb Color

The SpHb Color parameter allows you to specify the color of the SpHb indicator on the monitor.

The values for the SpHb Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

SpOC Color

The SpOC Color parameter allows you to specify the color of the SpOC indicator on the monitor.

The values for the SpOC Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

PVI Color

The PVI Color parameter allows you to specify the color of the PVI indicator on the monitor.

The values for the PVI Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

PI Color

The PI Color parameter allows you to specify the color of the PI indicator on the monitor.

The values for the PI Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

NIBP Color

The NIBP Color parameter allows you to specify the color of the NIBP indicator on the X Series monitor.

The values for the NIBP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Cyan

CO₂ Color

The CO₂ Color parameter allows you to specify the color of the CO₂ indicator and waveform on the X Series monitor.

The values for the CO2 Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Magenta

Resp Color

The Resp Color parameter allows you to specify the color of the Resp indicator on the X Series monitor.

The values for the Resp Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Magenta

Temp Color

The Temp Color parameter allows you to specify the color of the Temp indicator on the X Series monitor.

The values for the Temp Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	White

IBP Source P1 Color

The IBP Source P1 Color parameter allows you to specify the color of the IBP Source P1 indicator on the X Series monitor.

The values for the IBP Source P1 Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Red

IBP Source P2 Color

The IBP Source P2 Color parameter allows you to specify the color of the IBP Source P2 indicator on the X Series monitor.

The values for the IBP Source P2 Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Cyan

IBP Source P3 Color

The IBP Source P3 Color parameter allows you to specify the color of the IBP Source P3 indicator on the X Series monitor.

The values for the IBP Source P3 Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	White

IBP Source ABP Color

The IBP Source ABP Color parameter allows you to specify the color of the IBP Source ABP indicator on the X Series monitor.

The values for the IBP Source ABP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Green

IBP Source ART Color

The IBP Source ART Color parameter allows you to specify the color of the IBP Source ART indicator on the X Series monitor.

The values for the IBP Source ART Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Red

IBP Source AO Color

The IBP Source AO Color parameter allows you to specify the color of the IBP Source AO indicator on the X Series monitor.

The values for the IBP Source AO Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

IBP Source CVP Color

The IBP Source CVP Color parameter allows you to specify the color of the IBP Source CVP indicator on the X Series monitor.

The values for the IBP Source CVP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Cyan

IBP Source ICP Color

The IBP Source ICP Color parameter allows you to specify the color of the IBP Source ICP indicator on the X Series monitor.

The values for the IBP Source ICP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	White

IBP Source LAP Color

The IBP Source LAP Color parameter allows you to specify the color of the IBP Source LAP indicator on the X Series monitor.

The values for the IBP Source LAP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

IBP Source PAP Color

The IBP Source PAP Color parameter allows you to specify the color of the IBP Source PAP indicator on the X Series monitor.

The values for the IBP Source PAP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Yellow

IBP Source RAP Color

The IBP Source RAP Color parameter allows you to specify the color of the IBP Source RAP indicator on the X Series monitor.

The values for the IBP Source RAP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Blue

IBP Source UAP Color

The IBP Source UAP Color parameter allows you to specify the color of the IBP Source UAP indicator on the X Series monitor.

The values for the IBP Source UAP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Red

IBP Source UVP Color

The IBP Source UVP Color parameter allows you to specify the color of the IBP Source UVP indicator on the X Series monitor.

The values for the IBP Source UVP Color parameter are:

Possible Values	Green, Blue, Yellow, Red, White, Magenta, Cyan
Default	Blue

Device Info

The following options allows you to set specific information on the X Series unit upon purchase.

Unit ID

The Unit ID parameter allows you to set a unique identification number for the X Series unit.

The values for the Unit ID parameter are:

Possible Values	Alphanumeric
Default	<blank>

Department Name

The Department Name parameter allows you to set the name of the department using the X Series unit.

The values for the Department Name parameter are:

Possible Values	Alphanumeric
Default	<blank>

Contact Telephone

The Contact Telephone parameter allows you to set the telephone number of a contact for the user of the X Series unit.

The values for the Contact Telephone parameter are:

Possible Values	Alphanumeric
Default	<blank>

Display Setup

The following options allow you to set values related to display.

Display FiCO2

The Display FiCO2 parameter allows you to enable or disable FiCO2 monitoring.

The values for the Display FiCO2 parameter are:

Possible Values	Yes, No
Default	No

Screen Cycle Display Formats

This Screen Cycle Display Formats parameter allows you to set the screen cycle display

The values for the Screen Cycle Display Formats parameter are:

Possible Values	All Available, Skip Large Numerics
Default	All Available

Auto Pads

Pads automatically display on the Top Trace when patient impedance is detected.

The values for the Auto Pads parameter are:

Possible Values	Enabled/Disabled
Default	Enabled

Top Trace Default ECG Lead

The Top Trace Default ECG Lead parameter allows you to set the default ECG Lead cable for the top trace.

The values for the Top Trace Default ECG Lead parameter are:

Possible Values	Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
Default	Pads

Auto ECG Lead

The Auto ECG Lead parameter allows you to automatically display in the top trace the desired ECG leads when connected. This parameter is only available when Top Trace Default ECG Lead is set to Pads.

When the unit is turned on and a new case is created, the Auto ECG Lead feature will display Pads in the top trace by default. If an ECG connection to a patient is detected before a Pads patient connection, the top trace will be changed to the ECG lead selected in the Auto ECG Lead field. When an ECG connection to a patient is detected and the lead selected by the Auto ECG Lead field is not available, the selected lead will be displayed. This operation is only performed once for the duration of a case.

The values for the Auto ECG Lead parameter are:

Possible Values	Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
Default	II

Default ECG Leads

The Default ECG Lead parameter allows you to select up to 3 additional traces to display on startup, if no other monitoring devices are attached.

The second and third default ECG lead selections are grayed out and set to Not Assigned until the first default ECG lead section field is set to a specific lead. Setting one of the Default ECG Lead to Not Assigned will automatically reset all of the lower priority selections to Not Assigned.

The values for Default ECG Leads parameters are:

First Default Lead	
Possible Values	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
Default	Not Assigned

Second Default Lead	
Possible Values	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
Default	Not Assigned

Third Default Lead	
Possible Values	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
Default	Not Assigned

Enable NVG Mode

The Enable NVG Mode parameter allows you to enable or disable night vision goggle Mode.

The values for the Enable NVG Mode parameter are:

Possible Values	No, Yes
Default	No

Log Transfer to USB Device

The Log Transfer to USB Device parameter allows you to enable or disable transfer of log files to a USB device. The values for the Log Transfer to USB Device parameter are:

Possible Values	No, Yes
Default	No

Import/Export to USB Device

The Import/Export parameter allows you to enable or disable transfer of configuration files to a USB device. The values for the Import/Export to USB Device parameter are:

Possible Values	No, Yes
Default	No

Supervisor Code

The Supervisor Code parameter allows you to set the Supervisor Code needed to access the Supervisor Menu.

The values for the Supervisor Code parameter are:

Possible Values	Numeric
Default	Contact the Zoll Technical Support Department if you do not have the default supervisor code.

Log Clear Code

The Log Clear Code parameter allows you to set a user confirmation or a user confirmation and supervisor code to clear the disclosure log file.

Log Clear Restriction

If you set this parameter to **User confirmation** the user will be prompted with a confirmation dialog before clearing the disclosure log. If you set this parameter to **Passcode** the user will have to enter a passcode and then be prompted with a confirmation dialog before clearing the disclosure log.

The values for the Log Clear Restriction parameter are:

Possible Values	User confirmation, Passcode
Default	User confirmation

Log Clear Code

The values for the Log Clear Code parameter are:

Possible Values	Numeric
Default	1234

AED/Advisory Parameters

The AED/Advisory parameters allow you to view and modify functions related to using the unit for automatic external and advisory defibrillation. The main level parameters in this menu include:

- General
- Energy Protocol
- AED Mode

General

The following options allow you to set basic AED functions.

Power Up Mode


The Power Up Mode parameter allows you to set the mode in which the unit starts, or powers up.

The values for the Power Up Mode parameter are:

Possible Values	Manual, AED
Default	Manual

Restart Analysis After CPR

When this parameter is set to **Yes**, the analysis starts automatically after the CPR interval. When this parameter is set to **No**, the message *TO RESUME, PRESS "ANALYZE"* is displayed at the end of the CPR interval, but the analysis does not restart automatically.

 **Note:** When the Power Up Mode parameter is set to Manual, this parameter is **Yes** (Enabled).

The values for the Restart Analysis After CPR parameter are:

Possible Values	Yes, No
Default	Yes

Manual Mode Analysis

The Manual Mode Analysis parameter allows you to set the form of Manual Mode Analysis.

The values for the Manual Mode Analysis parameter are:

Possible Values	Single Analysis, Analysis/CPR Protocol
Default	Single Analysis

Enable Voice Prompts

The Enable Voice Prompts parameter allows you to specify when to use voice prompts.

The values for the Enable Voice Prompts parameter are:

Possible Values	Always, AED Mode Only
Default	Always

Disarm Timeout

The Disarm Timeout parameter allows you to set the time to disarm AED.

The values for the Disarm Timeout parameter are:

Possible Values	30 sec, 60 sec
Default	30 sec

CPR Time

The CPR Time parameter allows you to set the CPR time.

The values for the CPR Time parameter are:

Possible Values	1 min, 1.5 min, 2 min, 2.5 min, 3 min, 3.5 min, 4 min
Default	2 min

CPR Message After “No Shock Advised”

The CPR Message After “No Shock Advised” parameter allows you to set the message given after the “No Shock Advised” message.

The values for the CPR Message After “No Shock Advised” parameter are:

Possible Values	Perform CPR, If No Pulse, Perform CPR
Default	Perform CPR

CPR Message After Shock Delivered


The CPR Message After Shock Delivered parameter allows you to set the message given after the shock is delivered.

The values for the CPR Message After Shock Delivered parameter are:

Possible Values	Perform CPR, If No Pulse, Perform CPR
Default	Perform CPR

Enable SCE

Use the SCE parameter to enable/disable the Shock Conversion Estimator (SCE). SCE enhances CPR Analysis and the effectiveness of CPR by estimating the probability of whether or not the current ECG can successfully be converted by shock therapy. If the SCE determines that the probability that the shock will not convert the patient's heart rhythm to an organized rhythm, the unit issues the *NO SHOCK ADVISED* prompt, indicating that the preferred method of treatment for the patient should continue to be CPR. The unit displays the SCE results with the message *SHOCK ADVISED SCE* or *NO SHOCK ADVISED SCE*.

 **Note:** Changes you make to the Enable SCE parameter take effect after the unit has been off for at least two minutes.

The values for the Enable SCE parameter are:

Possible Values	Yes, No
Default	No

SCE Alter Adv Decision

The SCE Alter Adv Decision parameter determines how many shocks (after power up) are withheld (in favor of immediately resuming CPR) when a shockable rhythm is present but the Shock Conversion Estimator (SCE) indicates that the probability of successful defibrillation is low. The numerical value of this parameter (1, 2, 3, or 4) determines how many successive ECG analyses the unit will use SCE results for the Shock/ No Shock determination. For example, when the SCE Alter Adv Decision parameter is set to 2, the unit uses the SCE results for the first two analyses after the start of a new case. All subsequent rhythm analyses will base the shock/no shock determination only upon the presence or absence of VF or shockable VT.

i Important: Applying SCE logic to a number of early advisory analyses emphasizes the performance of CPR when a shockable rhythm is unlikely to convert to a spontaneous rhythm.

📄 Note: Changes you make to the SCE Alter Adv Decision parameter take effect after power up.

The values for the SCE Alter Adv Decision parameter are:

Possible Values	Off, 1, 2, 3, 4
Default	1

📄 Note: The SCE Alter Adv Decision counter resets when the patient mode is changed or the unit is off for at least two minutes and prior to the start of a new case.

CPR Analysis (RapidShock)

Use the CPR Analysis parameter to enable/disable RapidShock. When RapidShock is enabled, the unit analyzes the ECG rhythm during chest compressions and then briefly reconfirms the analysis result during pauses in compressions. This reduces the amount of time needed to determine whether shock therapy is necessary. RapidShock is available in AED mode of operation only or in both AED and Rescue Protocol modes of operation.

📄 Note: Changes you make to the CPR Analysis parameter take effect after the unit has been off for at least two minutes.

The values for the CPR Analysis parameter are:

Possible Values	AED Mode Only, AED and CPR Protocol, Disabled
Default	Enabled

Energy Protocol

The following options allow you to set the energy levels of the shocks given.

Adult Shock 1 Energy

The Adult Shock 1 Energy parameter allows you to set the energy level of the first shock given for adult patients.

The values for the Adult Shock 1 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	120 J

Adult Shock 2 Energy

The Adult Shock 2 Energy parameter allows you to set the energy level of the second shock given for adult patients.

The values for the Adult Shock 2 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	150 J

Adult Shock 3 Energy

The Adult Shock 3 Energy parameter allows you to set the energy level of the third shock given for adult patients.

The values for the Adult Shock 3 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	200 J

Pediatric Shock 1 Energy

The Pediatric Shock 1 Energy parameter allows you to set the energy level of the first shock given for pediatric patients.

The values for the Pediatric Shock 1 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	50 J

Pediatric Shock 2 Energy

The Pediatric Shock 2 Energy parameter allows you to set the energy level of the second shock given for pediatric patients.

The values for the Pediatric Shock 2 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	70 J

Pediatric Shock 3 Energy

The Pediatric Shock 3 Energy parameter allows you to set the energy level of the third shock given for pediatric patients.

The values for the Pediatric Shock 3 Energy parameter are:

Possible Values	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120J, 150 J, 200 J
Default	85 J

AED Mode

The following options allow you to set specific functions of the AED Mode.

Begin with CPR

The Begin with CPR parameter allows you to specify whether or not to begin with CPR.

The values for the Begin with CPR parameter are:

Possible Values	No, Yes
Default	No

Manual Mode Restriction

The Manual Mode Restriction parameter allows you to set the restriction for entering Manual Mode.

The values for the Manual Mode Restriction parameter are:

Possible Values	User Conformation, Passcode
Default	User Conformation

Manual Mode Passcode

The Manual Mode Passcode parameter allows you to set the Manual Mode Passcode.

The values for the Manual Mode Passcode parameter are:

Possible Values	Numeric
Default	

NIBP Monitoring

The NIBP Monitoring parameter allows you to enable or disable NIBP monitoring during AED.

The values for the NIBP Monitoring parameter are:

Possible Values	Disable, Enable
Default	Disable

CO2 Monitoring

The CO2 Monitoring parameter allows you to enable or disable CO2 monitoring during AED.

The values for the CO2 Monitoring parameter are:

Possible Values	Disable, Enable
Default	Disable

SpO2 Monitoring

The SpO2 Monitoring parameter allows you to enable or disable SpO2 monitoring during AED.

The values for the SpO2 Monitoring parameter are:

Possible Values	Disable, Enable
Default	Disable

Temperature Monitoring

The Temperature Monitoring parameter allows you to enable or disable temperature monitoring during AED.

The values for the Temperature Monitoring parameter are:

Possible Values	Disable, Enable
Default	Disable

12-Lead Monitoring

The 12-Lead Monitoring parameter allows you to enable or disable 12-Lead monitoring during AED.

The values for the 12-Lead Monitoring are:

Possible Values	Disable, Enable
Default	Disable

Allow User to End CPR to Analyze

The Allow User to End CPR to Analyze parameter allows you to specify whether or not to end CPR in order to analyze.

The values for the Allow User to End CPR to Analyze parameter are:

Possible Values	No, Yes
Default	Yes

CPR Parameters

The CPR parameters allow you to view and modify functions related to using the unit for CPR. The main level parameters in this menu include:

- AHA/ERC Guidelines
- CPR Time
- Enable Metronome
- Constant Metronome
- Enable CPR Prompts
- Display “Fully Release” Prompt
- Log CPR Metrics
- Auto Display Compression Trace

AHA/ERC Guidelines

The AHA/ERC Guidelines parameter allows you to specify which edition of the AHA/ERC Guidelines the unit follows.

The values for the AHA/ERC Guidelines parameter are:

Possible Values	2005, 2010, 2015
Default	2010

CPR Time

The CPR Time parameter allows you to set the CPR time.

The values for the CPR Time parameter are:

Possible Values	1 min, 1.5 min, 2 min, 2.5 min, 3 min, 3.5 min, 4 min
Default	2 min

Enable Metronome

This parameter allows you to select which modes you want the CPR metronome enabled: Never (no modes), Always (Manual, Analysis/CPR Protocol, and AED modes), and AED Mode Only. This parameter works in conjunction with the Constant Metronome parameter, and the mode in which the unit is operating determines how these parameters function together. The following paragraphs and "CPR Metronome Activity" on the facing page provide more detailed information.

Enable Metronome Set to Never

The metronome is disabled in all modes.

Enable Metronome Set to Always

When Enable Metronome is set to Always and Constant Metronome is Enabled, the metronome is on when CPR is detected, and off when CPR is idle in all modes.

When Enable Metronome is set to Always and Constant Metronome is Disabled:

- **With AHA/ERC 2005/2010 Guidelines** – If Enable Metronome is set to Always, the metronome is active only when the reported compression rate falls below 80 cpm during the CPR period. If the chest compression sensor is connected, the metronome is suspended after compressions have not been detected for at least 2 seconds.
- **With AHA/ERC 2015 Guidelines Enabled** – If Enable Metronome is set to Always, the metronome is only active when the reported compression rate falls above or below the 2015 guideline range of 100-120 cpm during the CPR period. If the chest compression sensor is connected, the metronome is suspended after compressions have not been detected for at least 2 seconds.
- **AED Mode** – Metronome is on at the start of CPR period.

Enable Metronome Set to AED Mode Only

When Enable Metronome is set to AED Mode Only and Constant Metronome is Enabled:

- **Manual Mode Analysis/CPR Protocol Mode** – No metronome.
- **AED Mode** – Metronome is on when CPR is detected, and off when CPR is idle.

When Enable Metronome is set to AED Mode Only and Constant Metronome is Disabled:

- **Manual Mode Analysis/CPR Protocol Mode** – No metronome.
- **AED Mode** – Metronome is on at the start of CPR period.

The values for the Enable Metronome parameter are:

Possible Values	Never, Always, AED Mode Only
Default	Never

CPR Metronome Activity

Enable Metronome Setting	AED Mode	Analysis/CPR Protocol Mode	Manual Mode
Constant Metronome Enabled			
AED Mode Only	On when CPR detected Off when CPR Idle	Off	Off
Always	On when CPR detected Off when CPR Idle	On when CPR detected Off when CPR Idle	On when CPR detected Off when CPR Idle
Never	Off	Off	Off
Constant Metronome Disabled			
AED Mode Only	On at start of CPR period	Off	Off
Always	On at start of CPR period	With AHA/ERC 2005 or 2010 Guidelines: On only at <80 compressions per minute. With AHA/ERC 2015 Guidelines: On when compression rate falls above or below 100-120 compressions per minute.	With AHA/ERC 2005 or 2010 Guidelines: On only at <80 compressions per minute. With AHA/ERC 2015 Guidelines: On when compression rate falls above or below 100-120 compressions per minute.
Never	Off	Off	Off

Constant Metronome

This parameter allows you to enable or disable the constant metronome. This parameter works in conjunction with the Enable Metronome parameter, and the mode in which the unit is operating (Manual, Analysis/CPR Protocol, or AED) determines how these parameters function together. When the Enabled Metronome parameter is set to Never, Constant Metronome is disabled. The following paragraphs and "CPR Metronome Activity" on the previous page provide more detailed information.

Constant Metronome Enabled

When Constant Metronome is Enabled and Enable Metronome is set to Always, the metronome is on when CPR is detected, and off when CPR is idle in all modes.

When Constant Metronome is Enabled and Enable Metronome is set to AED Mode Only:

- **Manual Mode and Analysis/CPR Protocol Mode** – No metronome.
- **AED Mode** – Metronome is on when CPR is detected, and off when CPR is idle.

Constant Metronome Disabled

When Constant Metronome is Disabled and Enable Metronome is set to Always:

- **Manual Mode and Analysis/CPR Protocol Mode** -
 - With AHA/ERC 2005 or 2010 Guidelines: The metronome is only on when compressions are less than 80 per minute during CPR.
 - With AHA/ERC 2015 Guidelines: The metronome is only on when compressions fall above or below the range of 100-120 per minute during the CPR.
- **AED Mode** – Metronome is on at the start of CPR period.

When Constant Metronome is Disabled and Enable Metronome is set to AED Mode Only:

- **Manual Mode and Analysis/CPR Protocol Mode** – No metronome.
- **AED Mode** – Metronome is on at the start of CPR period.

The values for the Constant Metronome parameter are:

Possible Values	Enabled, Disabled
Default	Disabled

Enable CPR Prompts

The Enable CPR Prompts parameter allows you to specify what CPR prompts are used. CPR prompts are only available when using Adult CPR electrodes.

The values for the Enable CPR Prompts parameter are:

Possible Values	Text and Voice, Text Only
Default	Text and Voice

Display “Fully Release” Prompt

The Display “Fully Release” Prompt parameter allows you to enable or disable the “Fully Release” prompt. The “Fully Release” text prompt is only available when using Adult CPR electrodes.

The values for the Display “Fully Release” Prompt parameter are:

Possible Values	No, Yes
Default	Yes

Log CPR Metrics

The Log CPR Metrics parameter allows you to specify whether or not to log CPR metrics.

The values for the Log CPR Metrics parameter are:

Possible Values	No, Yes
Default	Yes

Auto Display Compression Trace

This parameter allows you to enable or disable the CPR Compression Bar Graph. When this parameter is enabled (set to Yes), the CPR Compression Bar Graph automatically displays when compressions are detected. The CPR Compression Bar Graph is only available when using Adult CPR electrodes.

The values for the Auto Display Compression Trace parameter are:

Possible Values	Yes, No
Default	Yes

ResQCPR™

The ResQCPR parameter allows you to enable or disable the ResQCPR feature. The ResQCPR feature modifies Real CPR feedback for use with the ResQCPR system. When the ResQCPR parameter is enabled and the ResQCPR system is in use, a ResQCPR quick access key on the X Series unit allows you to switch from the CPR dashboard to the ResQCPR dashboard.

The values for the ResQCPR parameter are:

Possible Values	Disable, Enable
Default	Disable

Communications Parameters


The Communication parameters allow you to view and modify functions related to unit communications. The main level parameters in this menu include:

Menu	Description
Wireless	Use to enable or disable wireless capability. When disabled, all internal wireless capabilities are disabled in the unit.
Bluetooth	Use to enable or disable Bluetooth capability. When disabled, all Bluetooth capabilities are disabled in the unit.
Wi-Fi	Use to enable or disable Wi-Fi and to configure access point profiles. When disabled, all internal Wi-Fi capabilities are disabled in the unit. See "Configuring Access Point Profiles" on page 192.
Cellular	Use to configure USB cellular modem or Bluetooth devices. See "Setting up Cellular Communication."
Ethernet	Use to enable or disable Ethernet capability. See Setting up an Ethernet Connection to configure your Ethernet connection.
12-Lead Reports	Use to configure the server, and view or update distribution lists. Storage and management of 12-lead reports and full disclosure case files on the unit can be managed from third-party software such as RescueNet [®] . See Configuring 12-Lead Report Transmissions.
Disclosure Log	Use to configure the server to which to send the log data. The unit allows you to send the disclosure logs and audio for up to 15 cases at one time to a remote server. See Configuring Disclosure Log Transmissions.
Data Services	Use to set up a password that clients must use to access the unit. See Setting Passwords.
Clock Sync	Use to configure Clock Synchronization. The Clock Synchronization feature allows you to synchronize the unit's internal real time clock to an external reference time source using Simple Network Time Protocol. See Configuring Clock Synchronization.
Mobile Streaming	Use to stream waveforms and physiological parameters for the patient being monitored.

Menu	Description
Readiness Test	Use to configure the server to which to send Readiness Test log results and to enable/disable the automatic transmission of Readiness Test logs. When Readiness Test is enabled X Series unit performs a self-test and transmits the results to a ZOLL Defib Dashboard server.

Wireless

The Wireless parameter allows you to enable or disable wireless communications.


 **Note:** Disabling Wireless turns off all internal wireless capabilities.

The values for the Wireless parameter are:

Possible Values	Disabled, Enabled
Default	Enabled

Bluetooth

The Bluetooth parameter allows you to enable or disable Bluetooth communications.

 **Note:** This feature must be configured prior to use.

The values for the Bluetooth parameter are:

Possible Values	Disabled, Enabled
Default	Disabled

Wi-Fi

The WiFi parameter allows you to enable or disable WiFi communications, and if enabled, to configure access profiles.


 **Note:** This feature must be configured prior to use.

The values for the WiFi parameter are:

Possible Values	Disabled, Enabled
Default	Enabled
If Enabled	Configure

Power Management

The Power Management parameter allows you to enable/disable Wi-Fi power management. This parameter is located within the Configure Wi-Fi menu; to access, select uringure in the Wi-Fi parameter section when enabled.

 **Note:** This parameter is only applicable to X Series units with CP version 02.08.011.4084 andMP/PP version 02.18.03.00 or higher.


The values of the Power Management parameter are:


Possible Values	Disabled, Enabled
Default	Enabled

Configuring Access Point Profiles

To configure Wi-Fi Access Point Profiles:


1. Use the navigation keys to highlight and select **Wi-Fi Configure**. The Configure Wi-Fi dialog box displays.

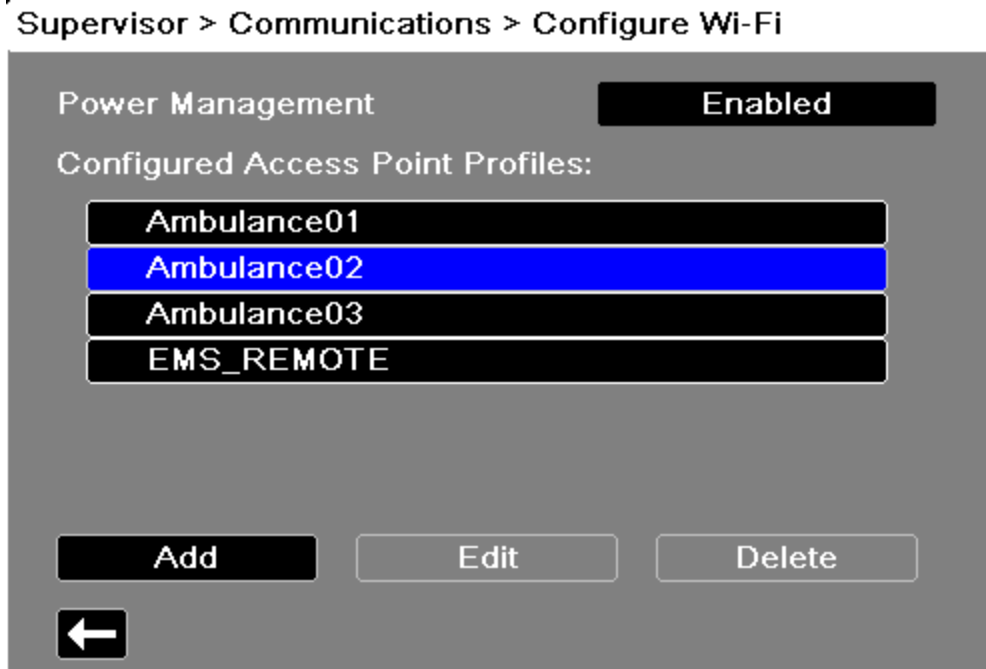

 **Note:** Selecting a profile in the list allows you to edit or delete the profile. It does not make a profile the active Wi-Fi profile selection.

2. If you would like to delete an existing profile use the navigation keys to select the desired profile (a green check mark identifies a selected profile) then navigate to and select **Delete**. After deleting a file, be sure to review the defined Wi-Fi profiles and update any which referred to the deleted file.
3. If you would like to:
 - Edit a profile use the navigation keys to select the profile (a green check mark identifies a selected profile) then navigate to and select **Edit**.
 - Add a new Access Point Profile, use the navigation keys to highlight and select **Add**.

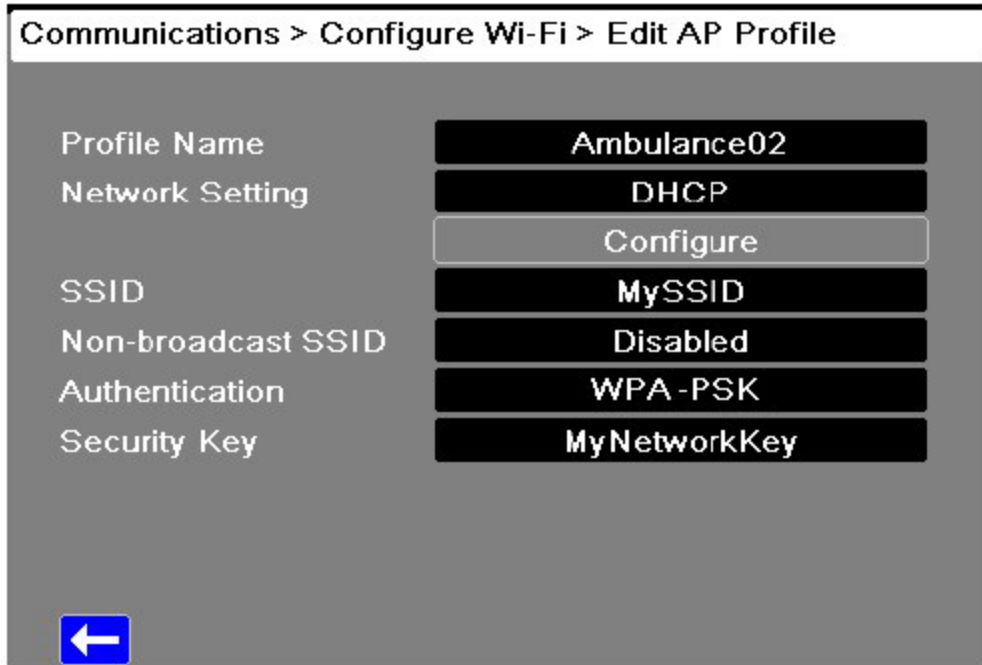

The Edit AP Profile dialog box displays.



4. Enter a new or modify an existing profile name by navigating to and selecting **Profile Name**. The Profile Name dialog displays. Use the alphabetical keypad and quick access keys to enter the profile name then press **SAVE**. You can press **Cancel** to return to the Configure Wi-Fi menu without saving changes.
5. Specify that this profile is either DHCP or a Static IP by using the navigation keys to select **Network Setting** then choosing either **DHCP** or **Static IP** as appropriate.
6. If you selected Static IP, navigate to and select **Network Setting > Configure**. The Configure Static IP dialog displays. Use the numeric keypad to enter values for the IP Address, Subnet Mask, Default Gateway, Preferred DNS Server, and Alternate DNS Server then press **Save**.
7. Provide the SSID name by navigating to and selecting **SSID**. The SSID dialog box displays. Use the alphabetical keypad and quick access keys to enter the SSID name then press **SAVE**.
8. Navigate to and select **Non-broadcast SSID** then specify whether to enable or disable connection to non-broadcast or hidden SSIDs by selecting **Enabled** or **Disabled** as appropriate. For Wi-Fi networks configured not to broadcast their SSID, you must set this field to **Enabled** in order to allow the X Series unit to connect to the Wi-Fi network.
9. Navigate to and select **Authentication** and then select the authentication type. The X Series unit supports three types of Wi-Fi authentication: WPA-PSK (Wi-Fi Protected Access, Pre-shared key), WPA2-PSK (Wi-Fi Protected Access II, Pre-shared key), and Enterprise.


10. When finished adding or editing WiFi profiles, press the back arrow (⬅️) to return to the list of configured access point profiles. Press ⬅️ to return to the Communications menu.

If you choose	Then
<p>WPA-PSK or WPA2-PSK</p>	<p>You must provide a security key. Navigate to and select Security Key. The Security Key dialog displays. Enter a 8-64 character security key by using the alphabetical keypad then press SAVE.</p>
<p>Authentication</p>	<p>Further configuration is required. Use the navigation keys to select Enterprise Settings then select Configure. The Enterprise Config dialog displays. Provide the Identity, and Authentication Protocol (TLS or PEAP). Some fields will be grayed out, depending on the Authentication Protocol that you choose.</p> <p>If you select PEAP (Protected Extensible Authentication Protocol), you must enter the Password corresponding to the Identity entered. If the network requires it, you may enter an Anonymous Identity. The X Series unit only supports PEAPv0/v1 + MSCHAPV2</p> <p>If you select TLS (Transport Layer Security), you must select a Client Certificate and enter the Private Key Password corresponding to the private key in the Certificate file.</p> <p>For TLS client certificates, you can choose from a list of pre-loaded files. The X Series unit only supports PFX format client certificates. To load a new certificate from a USB device:</p> <ol style="list-style-type: none"> a. Select Manage Client Certificates. A list of existing certificates displays. b. Select Load from USB. A list of files on the USB device displays. Filenames can only be 24 characters long. Filenames longer than 24 characters will not appear in the list. c. Use the arrow keys to select the desired file then select Load from USB. Only one file can be selected at a time. d. After pressing Load Selected, press the back arrow (⬅️) to view the list of all currently loaded certificates and keys.

If you choose	Then
	<p>e. Press  to return to the Enterprise Config menu, and  again to return to the Add Access Point Profile menu.</p> <p>The X Series provides pre-loaded certificate files for common certificate authorities. For organizations that act as their own CA or use a CA that the X Series does not provide, the X Series unit allows you to load your own CA certificates. To load a new CA certificate:</p> <ol style="list-style-type: none"> Insert the USB device with the CA certificates into the USB port on the right side of the unit. Select Load CA Certificate from USB. A list of files on the USB device displays. The X Series supports PEM, DER, and P7B CA certificate formats. Use the arrow keys to select the desired file then select Load Selected. Only one file can be selected at a time. When the Certificate Loaded dialog box displays, select OK to confirm.

Cellular

The Cellular parameter allows you to perform the configuration of cellular connectivity. This includes Bluetooth phone pairing operations, setup for cellular service providers, and selections for USB cellular modem usage.

 **Note:** This feature must be configured prior to use.


The values for the Cellular parameter are:

If Enabled	Configure
-------------------	-----------

You can configure your USB cellular modem or Bluetooth device to work with the X Series unit.

Under the Communications menu, select **Cellular Configure**. The Cellular menu appears.

Supervisor > Communications > Cellular

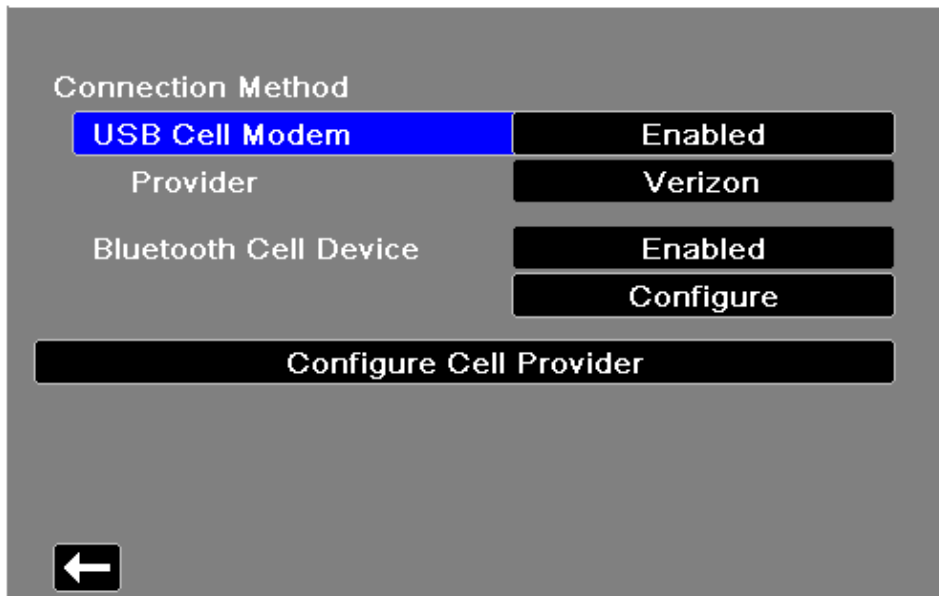

You can use the arrow keys to enable or disable all cellular functions, and configure your cell phone provider information.

Configuring Cell Phone Provider Information

You can create up to three cell phone providers. Use the arrow keys to enter information about your cell phone provider:


- **Provider Name** – Provider name is an identifier you choose to refer to this cell phone provider.
- **Call Number** – Call number is an access number sometimes referred to as a Dial String.
- **Account Name and Password** – Some providers require an account name and password. If your provider does not require these fields, leave them blank.

Configuring and Pairing Bluetooth Devices

Use Bluetooth devices with DUN or PAN-NAP profiles only. Not all cellular phones transmit reliably; contact your local ZOLL Medical Corporation sales representative for a list of approved devices.

To pair a Bluetooth device:


1. Select **Bluetooth Cell Device Configure** in the Cellular menu.
2. Use the navigation keys and select **New**. The unit scans for and lists nearby Bluetooth devices that are in discoverable mode.
3. Use the navigation keys to select the desired device. A green check mark indicates which device has been selected.

4. It may be necessary to change the PIN on the X Series unit. Check your device's documentation. To change the PIN, select **Change Pin** in the Pair New Device menu, then use the numerical keypad to enter the desired PIN.
5. Select **Pair Now** to pair the device.
6. On your Bluetooth device, accept the pairing request and enter the PIN, if prompted. The X Series unit then returns to the list of paired devices.
7. You can then edit the settings of the paired device by selecting **Edit** and giving the device an optional nickname or setting the provider. A provider is not required for Bluetooth devices using the PAN-NAP profile. Press the back arrow () to return to the list of paired devices.

Ethernet


You can use an Ethernet adapter to transfer data to a PC via an Ethernet cable.

1. In the Communications menu, select **Ethernet Configure**.
2. Use the navigation keys to select either **DHCP** or **Static IP**. If you select **Static IP**, use the numeric keypad to enter values for the IP Address, Subnet Mask, Default Gateway, Preferred DNS Server, and Alternate DNS Server.

 **Note:** If you are using an Ethernet adapter to transfer data to a PC, ZOLL recommends using the Static IP Address 169.254.1.250 and the Subnet Mask 169.254.1.250

12-Lead Reports

The Reports parameter allows you to configure the sending of 12-Lead reports. This includes configuring the RescueNet report server as well as customer account credentials.

 **Note:** This feature must be configured prior to use.


The values for the Reports parameter are:

If Enabled	Configure
-------------------	-----------

The device can send 12-lead reports to the ZOLL RescueNet software. Distribution List entries defined in your RescueNet account specify possible report destinations, each of which consists of one or many recipients such as email addresses and fax machines. You must have a RescueNet account set up before you can complete configuration of the device.

To configure 12-Lead reports transmissions:


1. In the Communications menu, select **12-Lead Reports Configure**.

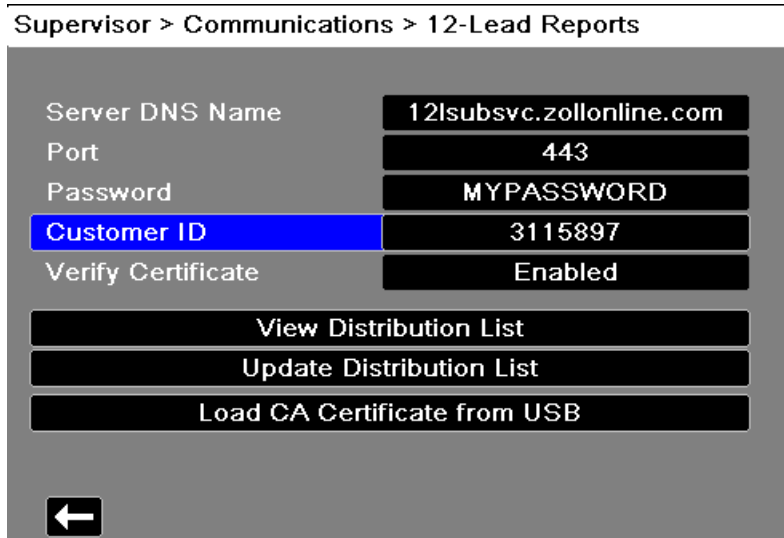

2. Enter your **RescueNet Customer ID** and **Password** in the fields provided.
3. In the **Verify Certificate** field specify whether or not you want to Enable or Disable (default) certificate verification for the RescueNet 12-lead server. If the **Verify Certificate** field is set to **Enabled**, when the X Series unit establishes a connection to the RescueNet 12-Lead Server it verifies the server's certificate and identity before the 12 Lead transmission can proceed. If the verification fails, the unit displays a status message indicating whether the failure was a result of certificate or identity validation. Enabling this option improves the security of the connection by confirming that the unit is communicating with the actual RescueNet server.



Note: The X Series comes with certificates file pre-loaded for common Certificate Authorities, including the CA used by the RescueNet server. All certificates, however, have expiration dates. It may be necessary to update the CA certificate in order to maintain the ability to successfully verify the RescueNet server's identity. Complete the following to update the CA certificate used to verify the RescueNet server:


- a. Insert a USB device with the certificates in the USB port on the right side of the X Series device.
 - b. Select the **Load CA Certificate from USB** field. The **Load from USB** window displays with a list of available Root CA certificate files.
 - c. In the **Files Available:** field select the desired certificate.
 - d. Select **Load Selected** field. A list of files on the USB device displays.
 - e. Use the arrow keys to select the desired certificate file then select **Load Selected**.
 - f. Press  to return to the 12-Lead Reports menu.
4. Before you can transmit 12-lead reports, you must download the current Distribution List for your account from the RescueNet server. If you have already configured your wireless preferences, you can use the **Update Distribution List** button to complete the download. The X Series retains the current Distribution List until a new one is downloaded. If you modify the Distribution List entries using RescueNet, you should download a new copy to the X Series device.
 5. When you are done entering data, press the back arrow to return to the Supervisor>Communications menu.

Disclosure Log

The X Series unit continuously stores information in a full disclosure case log for monitored patients. The unit can maintain up to 150 full disclosure cases that contain:

- Treatment events
- Trends
- ECG and other continuous waveforms
- Monitoring and event snapshots
- 12-lead snapshots and analysis.

For systems with a Disclosure Log server configured, the X Series unit allows you to wirelessly transmit the full Disclosure case logs and audio for up to 15 cases to a remote server for subsequent retrieval. The Disclosure Log parameter allows you to configure the remote server to which to wirelessly send case disclosure logs.


 **Note:** This feature must be configured prior to use.

The values for the Disclosure Log parameter are:

If Enabled	Configure
-------------------	-----------


To configure the remote server:

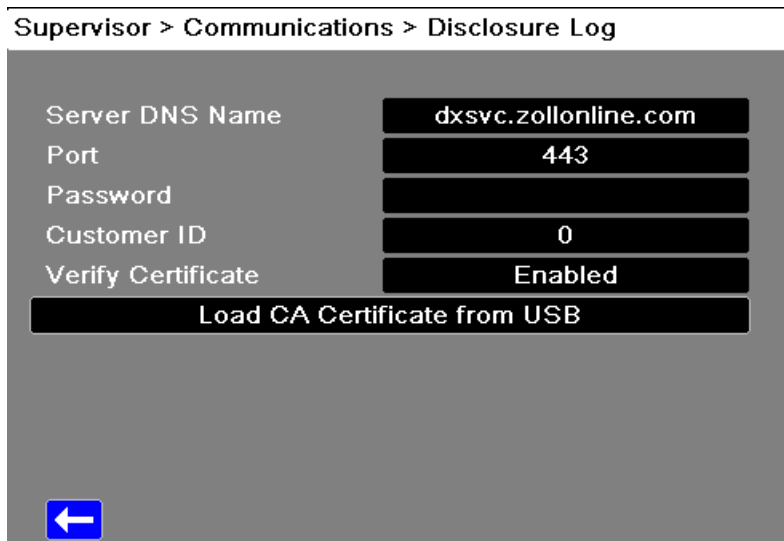

1. Under the Communications menu, select **Disclosure Log > Configure**. The Disclosure Log menu displays.



2. In the **Server DNS Name** field, enter the host name of the remote server to which to send the Log data.
3. Enter the **Port** to be used to connect to the server.
4. Enter your associated **Password** and **Customer ID** in the fields provided.

5. In the **Verify Certificate** field specify whether or not you want to Enable or Disable (default) certificate verification for the RescueNet server. If the **Verify Certificate** field is set to **Enabled**, when the X Series establishes a connection to the RescueNet server it verifies the server's certificate and identity before the Disclosure Log transmission can proceed. If the verification fails, the unit displays a status message indicating whether the failure was a result of certificate or identity validation. Enabling this option improves the security of the connection by confirming that the unit is communicating with the actual RescueNet server.

 **Note:** The X Series comes with certificates file pre-loaded for common Certificate Authorities, including the CA used by the RescueNet server. All certificates, however, have expiration dates. It may be necessary to update the CA certificate in order to maintain the ability to successfully verify the RescueNet server's identity. Complete the following to update the CA certificate used to verify the RescueNet server:

- a. Insert a USB device with the certificates in the USB port on the right side of the device.
 - b. Select the **Load CA Certificate from USB** field. The **Load from USB** window displays with a list of available Root CA certificate files.
 - c. In the **Files Available:** field select the desired certificate.
 - d. Select **Load Selected** field. A list of files on the USB device displays.
 - e. Use the arrow keys to select the desired certificate file then select **Load Selected**.
 - f. Press  to return to the Disclosure Log menu.
6. When you are done entering data, press the back arrow () to return to the Supervisor>Communications menu.

Data Services

For X Series devices interfacing with ePCR (electronic Patient Care Reporting) software, the Data Services parameter allows you to set the password necessary for clients to use before they can access data from the X Series device. Clients that fail to provide the correct password will be denied data access to the X Series.

From the Data Services menu you can:


- Set your own password.
- Automatically generate password.
- Reset current password back to the default.


i Important: For security ZOLL recommends setting your own password. When you set your own password you need to configure the same password in the ePCR software. Be sure the password you choose is sufficiently strong. Consider the following recommendations when setting passwords:

- Use a minimum of 16 characters.
- Use one uppercase and one lowercase character.
- Use one special character for example, !#\$%.
- Avoid patterns and character repetitions.
- Avoid common phrases or words associated with your institution.
- Avoid common passwords such as email and computer login passwords.
- Generate random passwords whenever possible.

To configure passwords:

1. Under the Communications menu, select **Data Services > Configure**. The Data Services menu displays.

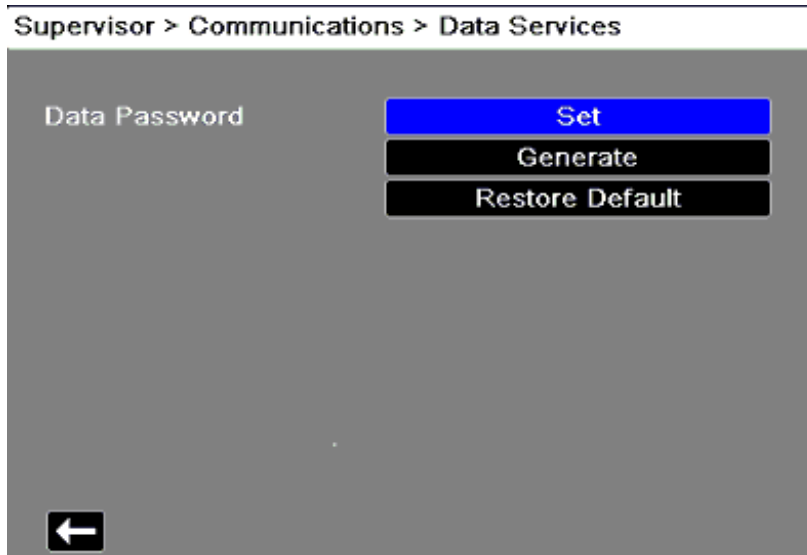

2. If you would like to create your own password:
 - a. Select the **Set** field. The Password menu displays.
 - b. Use the alphabetical keypad and quick access keys to specify a password of at least 8 characters in length. Be sure to make the password you supply is sufficiently strong. See password recommendations above.
 - c. Press **SAVE**. You can press **Cancel** to return to the Data Services menu without saving changes.
3. If you would like to have the X Series automatically generate a password for you:
 - a. Select the **Generate** field. The Password menu displays with a random password included.
 - b. Modify the password, if desired. Passwords must be at least 8 characters long. Be sure to make the password you supply is sufficiently strong. See password recommendations above.
 - c. Select **SAVE**.
4. If you would like to restore the password to its default:
 - a. Select the **Restore Default** field.
 - b. On the confirmation dialog box that displays, select **OK**.
5. When you are done with the Data Services menu press  to return to the Supervisor Communications menu.

Clock Sync

The Clock Synchronization feature allows you to synchronize the X Series unit's internal real time clock to an external reference time source using Simple Network Time Protocol (SNTP). SNTP servers provide Coordinated Universal Time (UTC) which is the primary time standard by which the world regulates clocks and time. In order to convert this standard time to the local time, the X Series allows you to configure a timezone offset. A timezone offset is the difference in hours and quarter-hours from UTC time for a particular locale. The timezone offset is a positive number if the local timezone is ahead of UTC and a negative number if it is behind.

Additionally, for locations that observe Daylight Saving Time, the X Series provides a setting that, when set to On, immediately applies an additional 1-hour positive offset from UTC. The supervisor sets the initial value and the operator may subsequently toggle Daylight Saving Time on or off as necessary on the Set Date and Time screen.

When Clock Sync is enabled and a network connection is available, the X Series unit automatically acquires a reference time from the SNTP server and converts the reference time to local time. The X Series will acquire a reference time from the SNTP server if no reference time has been acquired since the unit was powered on or the Clock Sync configuration settings have been changed. To ensure that the real time clock never changes during patient treatments, the X Series will make adjustments to its clock, if necessary, after the unit has been off for at least two minutes and prior to the start of a new case.

When Clock Sync is enabled, you can see the time of the last synchronization on the Set Date and Time screen. The X Series also provides the message *CLOCK SYNCHRONIZATION FAILED* when an attempt at synchronization fails after the SNTP server settings have changed or if there has been no successful synchronization for 1 week or more. See the X Series Operator's Guide for information on using the Set Date and Time screen.


To configure clock synchronization:

1. Under the Communications menu, select **Clock Sync > Configure**. The Clock Synchronization menu displays.

Supervisor > Communications > Clock Sync	
Clock Sync	SNTP
Time Server Name	TIME.EXAMPLE.COM
Time Server Port	123
Daylight Saving Time	Off
Time Zone Offset Hours	00
Time Zone Offset Mins.	00


- In the **Clock Sync** field, specify whether or not to synchronize the X Series internal real time clock to an external time source.
 - Select **SNTP** if you would like to synchronize the X Series clock to an external time source.
 - Select **None** (default) if you do not wish to synchronize the X Series internal real time clock to an external source.
- In the **Time Server Name** field, specify the time server to use for clock synchronization by either entering the host name or an IPv4 address for the server.
- In the **Time Server Port** field, enter a numeric UDP port number for connecting to the external server. The default is 123.
- In the **Daylight Saving Time** field, indicate whether to use Daylight Saving Time or not by selecting **On** or **Off** (default) as appropriate. Adjustments to Daylight Saving Time take effect immediately and must be manually adjusted at every transition between Standard Time and Daylight Saving Time. The Daylight Saving Time offset is zero if Daylight Saving Time is set to **Off** and 60 minutes if Daylight Saving Time is set to **On**.

Note: UTC time is a constant reference time. It does not change to reflect Daylight Saving Time or Summer Time. The Daylight Saving Time field allows you to accommodate locales that observe Daylight Saving Time or Summer Time. So, for example, UTC is 5 hours ahead of local time on the east coast of the United States during winter, but 4 hours ahead during summer.
- Indicate the time zone offset value to use for your particular locale in the **Time Zone Offset Hours** (default 0) and **Time Zone Offset Mins.** (default 0) fields as appropriate.

7. When you are done entering data, press the back arrow () to return to the Supervisor > Communications menu. When changes are made on the Clock Sync menu the X Series attempts to synchronize its time if a network connection is available. To ensure the real time clock never changes during patient treatments, any necessary clock synchronization corrections will not occur, however, until the unit has been off for at least two minutes and prior to the start of a new case.


Mobile Streaming

The Mobile Streaming parameter allows the X Series to stream waveforms and physiologic data for remote viewing of the patient being monitored. This information can be viewed by a remote user via the ZOLL RescueNet Live Client application.

 See the ZOLL *RescueNet® Live User Guide* for information on configuring the remote view server and viewing patient data remotely.

The values for the Mobile Streaming parameter are:

Possible Values	Disabled, Enabled
Default	Disabled
If Enabled	Configure

 When using the RescueNet Live feature the following configuration settings must be made for X Series devices connected in a wired configuration via an Ethernet switch, Wireless communication must be disabled. Conversely, Ethernet communication must be disabled for devices connected via a wireless connection. See "Wi-Fi" on page 190 and "Ethernet" on page 197 for information on enabling/disabling Wi-Fi and Ethernet communications.

 Ensure that **Continuous Waveform Recording** is enabled on the device, and **Record Displayed Waveforms** is set to **All Displayed Waveforms**.

To configure the X Series for remote view:

1. Enable RescueNet Live in the Communications menu by selecting **Mobile Streaming > Enabled**.
2. Select **Mobile Streaming > Configure**. The Mobile Streaming menu displays.
3. In the **Server DNS Name** field, enter the host name of the remote server.
4. Enter the **Port** to be used to connect to the server.
5. Enter your associated **Password** and **Customer ID** in the fields provided.

6. In the **Verify Certificate** field specify whether or not you want to **Enable** or **Disable** (default) certificate verification for the remote server. If the Verify Certificate field is set to **Enabled**, when the unit establishes a connection to the server it verifies the server's certificate and identity before proceeding. If the verification fails, the unit displays a status message indicating whether the failure was a result of certificate or identity validation. Enabling this option improves the security of the connection by confirming that the unit is communicating with the actual server.



Note: The unit comes with certificates file pre-loaded for common Certificate Authorities. All certificates, however, have expiration dates. It may be necessary to update the CA certificate in order to maintain the ability to successfully verify the server's identity. Complete the following to update the CA certificate used to verify the server:

- a. Insert a USB device with the certificates in the USB port on the right side of the unit.
 - b. Select the **Load CA Certificate from USB** field. The Load from USB window displays with a list of available Root CA certificate files.
 - c. In the **Files Available:** field select the desired certificate.
 - d. Select **Load Selected** field. A list of files on the USB device displays.
 - e. Use the arrow keys to select the desired certificate file then select **Load Selected**.
 - f. Press the back arrow to return to the Mobile Streaming menu.
7. When you are done with the Mobile Streaming menu press the back arrow to return to the Supervisor Communications menu.

Readiness Test

The Readiness Test parameter allows you to enable or disable the automatic transmission of a Readiness Test log and to configure the ZOLL Defib Dashboard server to which to transmit readiness logs. When Readiness Test is enabled, the unit automatically transmits the results of the Readiness Test to a ZOLL Defib Dashboard server. A readiness test log is created whenever a 30J test is performed and includes the results of the most recent self-test.




Note: You can manually transmit and print the results of the Readiness Test in the **Setup > Device Info** menu. See "Device Info" on page 22 for details.

To configure Readiness Test transmissions:

1. From the Communications menu, select Readiness Test > Configure. The Readiness Test screen displays.
2. Enable or disable the automatic transmission of a Readiness Test log to a configured server by selecting the **Auto Transmit Results** button then specifying **Enable** or **Disable** as appropriate.

3. In the **Server DNS Name** field, enter the host name of the remote server.
4. Enter the **Port** to be used to connect to the server.
5. Enter your associated **Password** and **Customer ID** in the fields provided.
6. In the **Verify Certificate** field, specify whether or not you want to **Enable** or **Disable** (default) certificate verification for the remote server. If the **Verify Certificate** field is set to **Enabled**, when the unit establishes a connection to the server it verifies the server's certificate and identity before proceeding. If the verification fails, the unit displays a status message indicating whether the failure was a result of certificate or identity validation. Enabling this option improves the security of the connection by confirming that the unit is communicating with the actual server.

 **Note:** The unit comes with certificate files pre-loaded for common Certificate Authorities. All certificates, however, have expiration dates. It may be necessary to update the CA certificate in order to maintain the ability to successfully verify the server's identity. Complete the following to update the CA certificate used to verify the server:
 - a. Insert a USB device with the certificates in the USB port on the right side of the unit.
 - b. Select the **Load CA Certificate from USB** field. The Load from USB window displays with a list of available Root CA certificate files.
 - c. In the **Files Available:** field select the desired certificate.
 - d. Select **Load Selected** field. A list of files on the USB device displays.
 - e. Use the arrow keys to select the desired certificate file then select **Load Selected**.
 - f. Press the back arrow to return to the Readiness Test screen.
7. When you are done with the Readiness Test screen press the back arrow to return to the Supervisor Communications menu.

Service Parameters

The Service parameters allow you to view and modify functions related to maintenance of the X Series unit. The main level parameters in this menu include:

- Device Tests
- Device Configuration
- Update/Upgrade
- Diagnostic
- Device Monitoring
- Research Options
- Service Code

Device Tests

The Device Tests are to be used to ensure that the X Series unit is operating at optimal efficiency.

Possible Tests	Keypad Test, LED Test, LCD Test, RFU Indicator Test, Audio Test, Printer Test, Battery Test, Fan Test, NIBP Test, USB Test
-----------------------	--

Device Configuration

The Device Configuration displays technical information about the X Series unit and its external equipment.

Update/Upgrade

The Update/Upgrade option allows you to make needed modifications to the X Series unit to keep it up to date.

Diagnostic

The following options allow you to set and view information on the X Series unit's performance.

ECG

The following options control values related to ECG diagnostics.

General > Resp Breath Marker

The Resp Breath Marker parameter allows you to enable or disable Resp Breath Marker.

The values for the Resp Breath Marker parameter are:

Possible Values	On, Off
Default	Off

Error Counts

This displays the number of ECG errors made by the unit.

NIBP

The following displays information related to NIBP diagnostics.

General

This displays basic information on the unit's NIBP equipment and software.

Calibration Info

This displays information on the unit's calibration software.

Reading Counts

This displays the number of readings related to NIBP monitoring.

Cycle/Error Counts

This display the number of errors related to NIBP monitoring.

NIBP Test

This allows you to test the NIBP system.

CO2

This displays information related to CO2 diagnostics.

SpO2

The following displays information related to SpO2 diagnostics.

Version Info

This displays technical SpO2 information on the current version of the X Series unit.

Error Counts

This displays the number of SpO2 errors made by the unit.

Temp/IBP

This displays information related to Temp/IBP diagnostics.

Error Counts

This displays the number of Temp/IBP errors made by the unit.

Defib/Pacer

The following displays information related to Defib/Pacer diagnostics.

Version Info

This displays technical Defib/Pacer information on the current version of the unit.

Cycle Counts

This displays the number of cycles made by the unit.

Error Counts

This displays the number of Defib/Pace errors made by the unit.

Communications

This allows you to securely transfer communication system logs in encrypted format to a USB device.

Device Monitoring

The Device Monitoring screen displays the number of time the X Series unit is used, the amount of time it has been on, and the number of times it went over temperature.

Research Options

The Waveform Recording Sample Rate parameter allows you to set the waveform recording sample rate.

Waveform Recording Sample Rate

The values for the Waveform Recording Sample Rate parameter are:

Possible Values	250 Hz, 500 Hz
Default	250 Hz

Service Code

The Service Code parameter allows you to set the Service Code.

The values for the Service Code parameter are:

Possible Values	Numeric
Default	<blank>

APPENDIX A

Configuration Tables

This appendix provides tables that show the range of configuration parameters and default values.

This appendix includes tables containing configuration parameters for the following menus:

- **Setup Menu** – Contains setup configuration options such as HR/PR tone, print speed, and trend display format.
- **Parameter Menus** – Contain configuration options for HR/PR, NIBP, Resp, SpO2, Temp, and IBP.
- **Supervisor Menu** – These menus contain default settings for alarms, treatment options, defibrillator/pacer, ECG, NIBP, printer, display, AED, CPR, communications, and service. Accessible only to authorized users.

Setup Menu

The following table includes the parameter ranges and default values for the Setup menu:

Menu	Submenu	Parameter Name	Default Value	Parameter Values
Setup	ECG	Filter Response	Monitor	Limited, Monitor
		HR/PR Tone	On	On, Off
		Resp Monitoring	On	On, Off
		Resp Lead	Lead II (RA-LL)	Lead I (RA-LA), Lead II (RA-LL)
		Patient Pace Marker	On	On, Off
		ECG Sweep Speed	25 mm/s	25 mm/s, 50 mm/s
	NIBP	NIBP Cuff Inflation Target	Adult: 160 Pediatric: 120 Neonate: 90	Adult: 120, 140, 160, 180, 200, 220, 240, 260 Pediatric: 80, 90, 100, 110, 120, 130, 140, 150 Neonate: 60, 70, 80, 90, 100, 110, 120, 130
	Display/Volume	Display Brightness	70%	10%, 20%, 30%, 40%, 50%, 60%, 70%, 80%, 90%, 100%
		HR/PR Tone	On	On, Off
		Volume	Medium	Low, Medium, High
	Printer	Print Speed	25 mm/s	25 mm/s, 50 mm/s
		Print Number of Traces	3	1, 2, 3, 4
		Print Grid	On	On, Off
		Print on Snapshot	Off	On, Off
		Print on Patient Alarm	Off	On, Off
		Print on NIBP	Off	On, Off
		Print on Trend	Off	On, Off

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	Trends	Trend Display Format	NIBP	Resp, SpO2, SpHb, Temp, P1, P2, P3, NIBP
		Trend Display Interval	5 min	30 sec, 1 min, 5 min, 10 min, 15 min, 30 min, 60 min
		Trend on Alarm	Off	On, Off
	Operational Tests	Keypad Test		
		LED Test		
		LCD Test		
		RFU Indicator Test		
		Audio Test		
		Printer Test		
	Device Info	Readiness Test		
		Log Review		
		Application Version		
Serial Number				
Unit ID				
Department Name				
Contact Telephone				

Parameter Menus

The following table includes parameter ranges and default values for the Parameter menus:

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
HR/PR	HR/PR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 120 Pediatric: 150 Neonate: 200	32-300
		Lower Limit	Adult: 50 Pediatric: 50 Neonate: 100	30-300
		Extreme Brady/Tachy Alarms	Tachy: +20 Brady: -10	Tachy: +5-+30 (by increments of 5) Brady: -5--30 (by increments of 5)
		LTA Monitoring	Off	On, Off
		HR/PR Tone	On	On, Off
		HR/PR Selected Source	ECG	ECG, SpO2, P1, P2, P3
		ECG Sweep Speed	25 mm/s	25 mm/s, 50 mm/s

Menu	Submenu	Parameter Name	Default Value	Parameter Values
NIBP	NIBP Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 220 Pediatric: 145 Neonate: 100	Adult: 32-260 Pediatric: 32-160 Neonate: 22-120
		Lower Limit	Adult: 75 Pediatric: 75 Neonate: 50	Adult: 30-258 Pediatric: 30-158 Neonate: 20-118
	NIBP Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 110 Pediatric: 100 Neonate: 70	Adult: 22-220 Pediatric: 22-130 Neonate: 12-110
		Lower Limit	Adult: 35 Pediatric: 35 Neonate: 30	Adult: 20-218 Pediatric: 20-128 Neonate: 10-108

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	NIBP MAP Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 120 Pediatric: 110 Neonate: 80	Adult: 25-230 Pediatric: 25-140 Neonate: 15-110
		Lower Limit	Adult: 50 Pediatric: 35 Neonate: 35	Adult: 23-228 Pediatric: 23-138 Neonate: 13-108
		NIBP Mode	Manual	Manual, Auto
		NIBP Auto Mode Interval	5 min	1 min, 2 min, 3 min, 5 min, 10 min, 15 min, 30 min, 60 min
		SmartCuf On/Off	On	On, Off

Menu	Submenu	Parameter Name	Default Value	Parameter Values
Resp	RR/BR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 50 Pediatric: 50 Neonate: 80	5-150
		Lower Limit	Adult: 3 Pediatric: 3 Neonate: 12	0-145
		No Breath Alarm	Adult: Off Pediatric: Off Neonate: Off	Adult: Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec Pediatric: Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec Neonate: Off, 10 sec, 15 sec, 20 sec
		Resp Monitoring	On	On, Off
		Resp Lead	Lead II (RA-LL)	Lead I (RA-LA), Lead II (RA-LL)
		CO2/Resp Sweep Speed	12.5 mm/s	3.13 mm/s, 6.25 mm/s, 12.5 mm/s

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
SpO2	SpO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 100 Pediatric: 100 Neonate: 95	52-100
		Lower Limit	85	50-98

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	SpCO Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 10 Pediatric: 10 Neonate: 10	2-100
		Lower Limit	Adult: 0 Pediatric: 0 Neonate: 0	0-98
	SpMet Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 3.0 Pediatric: 3.0 Neonate: 3.0	2.0, 2.5, 3.0, 3.5, 4.0, 4.5, 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5, 10.0-100.0
		Lower Limit	Adult: 0.0 Pediatric: 0.0 Neonate: 0.0	0.0, 0.5, 1.0, 1.5, 2.0, 2.5, 3.0, 3.5, 4.0, 4.5, 5.0, 5.5, 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0, 9.5, 10.0-98.0

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	SpHb Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 17.0 gdL 11.0 mmol/L Pediatric: 17.0 gdL 11.0 mmol/L Neonate: 17.0 gdL 11.0 mmol/L	gdL: 2 to 25 mmol/L: 2 to 15.5
		Lower Limit	Adult: 7.0 gdL 4.0 mmol/L Pediatric: 7.0 gdL 7.0 mmol/L Neonate: 7.0 gdL 7.0 mmol/L	gdL: 0 to 24.9 mmol/L: 0 to 15.4

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	SpOC Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 25 ml/dL Pediatric: 25 ml/dL Neonate: 25 ml/dL	0.2 to 35 ml/dL
		Lower Limit	Adult: 10 ml/dL Pediatric: 10 ml/dL Neonate: 10 ml/dL	0 to 34.9 ml/dL
	PVI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 40% Pediatric: 40% Neonate: 40%	2 to 100%
		Lower Limit	Adult: 5% Pediatric: 5% Neonate: 5%	0 to 98%

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	PI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	20%	0.2 to 20%
		Lower Limit	0%	0 to 19.8%
	SpCO Monitoring		Off	On, Off
	SpMet Monitoring		Off	On, Off
	SpHb Monitoring		Off	On, Off
	SpOC Monitoring		Off	On, Off
	PVI Monitoring		Off	On, Off
	PI Monitoring		Off	On, Off
	Averaging Time		8 seconds	4 seconds, 8 seconds, 16 seconds
	Sensitivity		Normal	Normal, Maximum
	SPHB Venous Mode		Off	On, Off
	SpHb Averaging Time		Long	Short, Medium, Long
HR/PR Tone		On	On, Off	

Menu	Submenu	Parameter Name	Default Value	Parameter Values
Temp	T1 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0
	T2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0
	ΔT Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 5.0 °C: 2.8	°F: 0.2-90.0 °C: 0.2-50.0
		Lower Limit	°F: 0.0 °C: 0.0	°F: 0.0-89.8 °C: 0.0-49.8
		T1 Source Label	T1	T1, ART, CORE, ESOPH, RECT, SKIN, VEN, NASO
		T2 Source Label	T2	T2, ART, CORE, ESOPH, RECT, SKIN, VEN, NASO

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
IBP1	P1 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 220 Pediatric: 145 Neonate: 100	-28-300
		Lower Limit	Adult: 75 Pediatric: 75 Neonate: 50	-30-298
	P1 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 110 Pediatric: 100 Neonate: 70	-28-300
		Lower Limit	Adult: 35 Pediatric: 35 Neonate: 30	-30-298

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	P1 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 120 Pediatric: 110 Neonate: 80	-28-300
		Lower Limit	Adult: 50 Pediatric: 50 Neonate: 35	-30-298
		Source Label	P1	P1, ABP, AO, ART, BAP, CVP, FAP, ICP, LAP, PAP, RAP, UAP, UVP
		Display Format	S/D (M)	S/D (M), S/D, (M) S/D, (M)

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
IBP2	P2 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 220 Pediatric: 145 Neonate: 100	-28-300
		Lower Limit	Adult: 75 Pediatric: 75 Neonate: 50	-30-298

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	P2 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 110 Pediatric: 100 Neonate: 70	-28-300
		Lower Limit	Adult: 35 Pediatric: 35 Neonate: 30	-30-298
	P2 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 120 Pediatric: 110 Neonate: 80	-28-300
		Lower Limit	Adult: 50 Pediatric: 50 Neonate: 35	-30-298
		Source Label	P2	P2, ABP, AO, ART, BAP, CVP, FAP, ICP, LAP, PAP, RAP, UAP, UVP
		Display Format	S/D (M)	S/D (M), S/D, (M) S/D, (M)

APPENDIX A Configuration Tables

Menu	Submenu	Parameter Name	Default Value	Parameter Values
IBP3	P3 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 220 Pediatric: 145 Neonate: 100	-28-300
		Lower Limit	Adult: 75 Pediatric: 75 Neonate: 50	-30-298
	P3 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 110 Pediatric: 100 Neonate: 70	-28-300
		Lower Limit	Adult: 35 Pediatric: 35 Neonate: 30	-30-298

Menu	Submenu	Parameter Name	Default Value	Parameter Values
	P3 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	Adult: 120 Pediatric: 110 Neonate: 80	-28-300
		Lower Limit	Adult: 50 Pediatric: 50 Neonate: 35	-30-298
		Source Label	P3	P3, ABP, AO, ART, BAP, CVP, FAP, ICP, LAP, PAP, RAP, UAP, UVP
		Display Format	S/D (M)	S/D (M), S/D, (M) S/D, (M)

Supervisor Menu

The following table includes parameter ranges and default values for the Supervisor menu:

		Parameter Name	Default Value	Parameter Values
Alarms	General	Default Patient Mode	Adult	Adult, Pediatric, Neonate
		Volume	Medium	Low, Medium, High
		Audio Paused Time (Alarm Suspend Time)	4 min	Disable, 2 min, 4 min, 15 min, Always On
		Alarm Latching Enable	No	Yes, No
		Reminder Alarm	Disable	Enable, Disable
		Reminder Alarm Interval	5 min	5 min, 10 min, 15 min
	Default Adult>LTA Monitoring	LTA Monitoring	Off	On, Off

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Adult>HR/PR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	120	32-300
		Lower Limit	50	30-298
	Default Adult>cl Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	220	32-260
		Lower Limit	75	30-258
	Default Adult>NIBP Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	22-220
		Lower Limit	35	20-218
	Default Adult>NIBP MAP Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	120	25-230
		Lower Limit	50	23-228

		Parameter Name	Default Value	Parameter Values
	Default Adult>EtCO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	60	5-150
		Lower Limit	8	0-145
	Default Adult>FiCO2 Alarm	Upper Limit Enable	Off	On, Off
		Upper Limit	8	2-98
	Default Adult>RR/BR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	50	5-150
		Lower Limit	3	0-145

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Adult>No Breath Alarm	No Breath Alarm	Off	Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec

		Parameter Name	Default Value	Parameter Values
	Default Adult>SpO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	52-100
		Lower Limit	85	50-98
	Default Adult>SpCO Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	10	2-100
		Lower Limit	0	0-98
	Default Adult>SpMet Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	3.0	2.0-100.0
		Lower Limit	0.0	0.0-98.0
	Default Adult>SpHb Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	17.0 gdl 11.0 mmol/L	gdL: 2 to 25 mmol/L: 2 to 15.5
		Lower Limit	7.0 gdl 4.0 mmol/L	gdL: 0 to 24.9 mmol/L: 0 to 15.4
Lower Limit Enable		Off	On, Off	

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Adult>SpOC Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	25 ml/dL	0.1 to 35 ml/dL
		Lower Limit	10 ml/dL	0 to 34.9 ml/dL
	Default Adult>PVI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	40%	2 to 100%
		Lower Limit	5%	0 to 98%
	Default Adult>PI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	20%	0.2 to 20%
		Lower Limit	0%	0 to 19.8%
	Default Adult>T2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0

		Parameter Name	Default Value	Parameter Values
	Default Adult>DT Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 5.0 °C: 2.8	°F: 0.2-90.0 °C: 0.2-50.0
		Lower Limit	°F: 0.0 °C: 0.0	°F: 0.0-89.8 °C: 0.0-49.8
	Default Adult>P1 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	220	-28-300
		Lower Limit	75	-30-298
	Default Adult>P1 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	35	-30-298

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Adult>P1 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	120	-28-300
		Lower Limit	50	-30-298
	Default Adult>P2 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	220	-28-300
		Lower Limit	75	-30-298
	Default Adult>P2 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	35	-30-298
	Default Adult>P2 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	120	-28-300
		Lower Limit	50	-30-298

		Parameter Name	Default Value	Parameter Values
	Default Adult>P3 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	220	-28-300
		Lower Limit	75	-30-298
	Default Adult>P3 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	35	-30-298
	Default Adult>P3 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	120	-28-300
		Lower Limit	50	-30-298
	Default Pediatric>LTA Monitoring	LTA Monitoring	Off	On, Off
	Default Pediatric> HR/PR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	150	32-300
Lower Limit		50	30-298	

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>NIBP Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	145	32-160
	Default Pediatric>NIBP Systolic Alarms	Lower Limit	75	30-158
	Default Pediatric>NIBP Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	22-130
		Lower Limit	35	20-128
	Default Pediatric>NIBP MAP Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	25-140
		Lower Limit	50	23-138
	Default Pediatric>EtCO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	60	5-150
		Lower Limit	8	0-145
Default Pediatric>FiCO2 Alarm	Upper Limit Enable	Off	On, Off	
	Upper Limit	8	2-98	

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>RR/BR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	50	5-150
		Lower Limit	3	0-145
	Default Pediatric>No Breath Alarm	No Breath Alarm	Off	Off, 10 sec, 15 sec, 20 sec, 25 sec, 30 sec, 40 sec, 50 sec, 60 sec
	Default Pediatric>SpO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	52-100
		Lower Limit	85	50-98
	Default Pediatric>SpCO Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	10	2-100
		Lower Limit	0	0-98
	Default Pediatric>SpMet Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	3	2.0-100.0
Lower Limit		0	0.0-98.0	

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>T2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0
	Default Pediatric>DT Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 5.0 °C: 2.8	°F: 0.2-90.0 °C: 0.2-50.0
		Lower Limit	°F: 0.0 °C: 0.0	°F: 0.0-89.8 °C: 0.0-49.8

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>P1 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	145	-28-300
		Lower Limit	75	-30-298
	Default Pediatric>P1 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	35	-30-298
	Default Pediatric>P1 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	50	-30-298
	Default Pediatric>P2 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	145	-28-300
		Lower Limit	75	-30-298

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>P2 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	35	-30-298
	Default Pediatric>P2 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	50	-30-298
	Default Pediatric>P3 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	145	-28-300
		Lower Limit	75	-30-298
	Default Pediatric>P3 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	35	-30-298

		Parameter Name	Default Value	Parameter Values
	Default Pediatric>P3 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	110	-28-300
		Lower Limit	50	-30-298
	Default Neonate>HR/PR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	200	32-300
		Lower Limit	100	30-298
	Default Neonate>NIBP Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	22-120
		Lower Limit	50	20-118
	Default Neonate>NIBP Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	70	12-110
		Lower Limit	30	10-108

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Neonate>NIBP MAP Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	80	15-110
		Lower Limit	35	13-108
	Default Neonate>EtCO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	60	5-150
		Lower Limit	8	0-145
	Default Neonate>FiCO2 Alarm	Upper Limit Enable	Off	On, Off
		Upper Limit	8	2-98

		Parameter Name	Default Value	Parameter Values
	Default Neonate>RR/BR Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	80	5-150
		Lower Limit	12	0-145
	Default Neonate>No Breath Alarm	No Breath Alarm	Off	Off, 10 sec, 15 sec, 20 sec
	Default Neonate>SpO2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	95	52-100
		Lower Limit	85	50-98
	Default Neonate>SpCO Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	10	2-100
		Lower Limit	0	0-98
	Default Neonate>SpMet Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	3	2.0-100.0
Lower Limit		0	0.0-98.0	

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Neonate>SpHb Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	17.0 gdL 11.0 mmol/L	gdL: 2 to 25 mmol/L: 2 to 15.5
		Lower Limit	7.0 gdL 4.0 mmol/L	gdL: 0 to 24.9 mmol/L: 0 to 15.4
	Default Neonate>SpOC Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	25 ml/dL	0.1 to 35 ml/dL
		Lower Limit	10 ml/dL	0 to 34.9 ml/dL
	Default Neonate>PVI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	40%	2 to 100%
		Lower Limit	5%	0 to 98%
	Default Neonate>PI Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	20%	0.2 to 20%
		Lower Limit	0%	0 to 19.8%
Default Neonate>T1 Alarm	Upper Limit Enable	Off	On, Off	
	Lower Limit	Off	On, Off	

		Parameter Name	Default Value	Parameter Values
		Enable		
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0
	Default Neonate>T2 Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 100.0 °C: 37.8	°F: 34.0-122.0 °C: 2.0-50.0
		Lower Limit	°F: 95.0 °C: 35.0	°F: 32.0-120.0 °C: 0.0-48.0
	Default Neonate>DT Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	°F: 5.0 °C: 2.8	°F: 0.2-90.0 °C: 0.2-50.0
		Lower Limit	°F: 0.0 °C: 0.0	°F: 0.0-89.8 °C: 0.0-49.8
	Default Neonate>P1 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	50	-30-298

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Default Neonate>P1 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	50	-30-298
	Default Neonate>P1 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	80	-28-300
		Lower Limit	35	-30-298
	Default Neonate>P2 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	50	-30-298
	Default Neonate>P2 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	70	-28-300
		Lower Limit	30	-30-298

		Parameter Name	Default Value	Parameter Values
	Default Neonate>P2 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	80	-28-300
		Lower Limit	35	-30-298
	Default Neonate>P3 Systolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	100	-28-300
		Lower Limit	50	-30-298
	Default Neonate>P3 Diastolic Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	70	-28-300
		Lower Limit	30	-30-298
	Default Neonate>P3 Mean Alarm	Upper Limit Enable	Off	On, Off
		Lower Limit Enable	Off	On, Off
		Upper Limit	80	-28-300
		Lower Limit	35	-30-298
Log	Treatment Options	Configuration Method	Select From Predefined List	Select From Pre-defined List, Define Custom Labels

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Treatment Button #1	O2	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
		Treatment Button #2	ASA	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
		Treatment Button #3	Nitro	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
		Treatment Button #4	Morph	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
		Treatment Button #5	IV	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate
		Treatment Button #6	B Block	O2, ASA, Nitro, Morph, IV, B Block, Lido, MgSO4, Valium, Sedate

		Parameter Name	Default Value	Parameter Values
Defib/Pacer	Default Settings	Adult Defib Ext. Pdl. Energy	120 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Pediatric Defib Ext. Pdl. Energy	50 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Neonate Defib Ext. Pdl. Energy	50 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Adult Defib Int. Pdl. Energy	10 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J
		Pediatric Defib Int. Pdl. Energy	10 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J
		Neonate Defib Int. Pdl. Energy	10 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J
		Sync After Cardioversion	Off	On, Off
		Default Pacer Mode	Demand	Demand, Fixed
		Default Pacer Rate	70	30-180 (<100 by increments of 5, >100 by increments of 10)
		Auto Display Pads/Paddles	No	Yes, No

		Parameter Name	Default Value	Parameter Values
	Energy Escalation	Basic Auto Energy Selection	Disabled	Disabled, Enabled

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Adult Shock 1 Energy	120 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Adult Shock 2 Energy	150 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Adult Shock 3 Energy	200 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Pediatric Shock 1 Energy	50 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Pediatric Shock 2 Energy	70 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J
		Pediatric Shock 3 Energy	85 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J, 85 J, 100 J, 120 J, 150 J, 200 J

		Parameter Name	Default Value	Parameter Values
ECG	General	AC Mains Filter	60 Hz	Off, 50 Hz, 60 Hz
		Auto Add Filter ECG	No	Yes, No
	12 Lead ECG	Print on 12-Lead Acquire	Off	On, Off
		Print Copies	1	1, 2, 3, 4, 5
		12-Lead Format	3x4	2x6, 3x4, 3x4 Cabrera, 3x4 Hybrid, 3x4 Cabrera Hybrid
		Print 10s Trace 1	I	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
		Print 10s Trace 2	Off	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
		Print 10s Trace 3	Off	Off, I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6
		12-Ld Frequency Response	Filtered Diagnostic	Diagnostic, Filtered Diagnostic
		12-Ld Analysis Enabled	On	On, Off
		Interpretive Text	On	On, Off
		Patient Information Prompt	On	On, Off
		Print 12-Ld 1mV Bracket	No	Yes, No

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
NIBP		Default NIBP Mode	Manual	Manual, Auto
		Default Auto Mode Interval	5 min	1 min, 2 min, 3 min, 5 min, 10 min, 15 min, 30 min, 60 min
		Default Cuff Inflation Target (Adult)	160	120-260 (by increments of 20)
		Default Cuff Inflation Target (Ped.)	120	80-150 (by increments of 10)
		Default Cuff Inflation Target (Neo.)	90	60-130 (by increments of 10)
Printer		Print on Defib/Pacer Events	On	On, Off

		Parameter Name	Default Value	Parameter Values
		Print on Event	Off	On, Off
		Print on Treatment Snapshot	Off	On, Off
		Print Treatment Summary Snapshots	On	On, Off
		Print on Analysis Results	Off	On, Off
		Print on Presenting Rhythm	Off	On, Off
		Print on Readiness Test Results	Off	On, Off

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
Display/Configuration	Localization	Language	English	Arabic (Voice only), Brazilian Portuguese, Simplified Chinese, Traditional Chinese, Czech, Danish, Dutch, English, Finnish, French, French Canadian, German, Italian, Japanese, Korean (Voice only), Norwegian, Polish, Romanian, Russian, Spanish, Swedish, Turkish

		Parameter Name	Default Value	Parameter Values
		Decimal Format	. (dot)	. (dot), , (comma)

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Date Format	MM/DD/YYYY	MM/DD/YYYY, DD/MM/YYYY, YYYY/MM/DD, MM-DD-YYYY, DD-MM-YYYY, YYYY-MM-DD
		Time Format	12 Hour, 24 Hour	24 Hour
		Temp Units	°F	°F, °C
		IBP Units	mmHg	mmHg, kPa
		CO2 Units	mmHg	mmHg, kPa,%
		NIBP Units	mmHg	mmHg, kPa
		ECG Lead Labels	AAMI	AAMI, IEC
		NIBP Display Format	S/D (M)	S/D (M), (M) S/D
		Length Units	Inch	Inch, cm
		AC Mains Filter	60 Hz	Off, 50 Hz, 60 Hz
		SpHb Units	g/dL, mmol/L	g/dL
	Color	ECG Color	Green	Green, Blue, Yellow, Red, White, Magenta, Cyan
		SpO2 Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		SpCO Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan

		Parameter Name	Default Value	Parameter Values
		SpMet Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		SpHb Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		SpOC Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		PVI Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		PI Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		NIBP Color	Cyan	Green, Blue, Yellow, Red, White, Magenta, Cyan

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		CO2 Color	Magenta	Green, Blue, Yellow, Red, White, Magenta, Cyan
		Resp Color	Magenta	Green, Blue, Yellow, Red, White, Magenta, Cyan
		Temp Color	White	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source P1 Color	Red	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source P2 Color	Cyan	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source P3 Color	White	Green, Blue, Yellow, Red, White, Magenta, Cyan

		Parameter Name	Default Value	Parameter Values
		IBP Source ABP Color	Green	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source ART Color	Red	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source AO Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source CVP Color	Cyan	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source ICP Color	White	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source LAP Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		IBP Source PAP Color	Yellow	Green, Blue, Yellow, Red, White, Magenta, Cyan

		Parameter Name	Default Value	Parameter Values
		IBP Source RAP Color	Blue	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source UAP Color	Red	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source UVP Color	Blue	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source BAP Color	Red	Green, Blue, Yellow, Red, White, Magenta, Cyan
		IBP Source FAP Color	Green	Green, Blue, Yellow, Red, White, Magenta, Cyan
	Clock	Daylight Saving Mode	Manual, Auto	Manual
		Configure		
		Current Time	DST, Standard	Standard
	Device Info	<blank>	Alphanumeric	
	Unit ID			
		Department Name	<blank>	Alphanumeric
		Contact Telephone	<blank>	Alphanumeric
	Display Setup	Display FiCO2	No	No, Yes

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Screen Cycle Display Formats	All Available	All Available, Skip Large Numerics
		Top Trace Default ECG Lead	Pads	Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
		Auto ECG Lead	II	Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
		Default ECG Lead	Pads	Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
		Default ECG Leads	Not Assigned	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
			Not Assigned	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
			Not Assigned	Not Assigned, Pads, I, II, III, aVR, aVL, aVF, V, V1, V2, V3, V4, V5, V6
		Enable NVG Mode	No	No, Yes

		Parameter Name	Default Value	Parameter Values
	Supervisor Code		Contact the Zoll Technical Support Department if you do not have the default supervisor code	Numeric
	Log Clear Code	Log Clear Restriction	Passcode	User confirmation, Passcode
		Log Clear Code	1234	Numeric

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
AED	General	Power Up Mode	Manual	Manual, AED
		Restart Analysis After CPR	No	Yes, No
		Manual Mode Analysis	Single Analysis	Single Analysis, Analysis/ CPR Protocol
		Enable Voice Prompts	Always	Always, AED Mode Only
		Disarm Timeout	30 sec	30 sec, 60 sec
		CPR Time	2 min	1 min, 1.5 min, 2 min, 2.5 min, 3 min, 3.5 min, 4 min
		CPR Message After "No Shock Advised"	Perform CPR	Perform CPR, If No Pulse, Perform CPR
		CPR Message After Shock Delivered	Perform CPR	Perform CPR, If No Pulse, Perform CPR
		Enable SCE	No	Yes, No
		SCE Alter Adv Decision	1	Off, 1, 2, 3, 4
		CPR Analysis	Disabled	AED Mode Only, AED and CPR Protocol, Disabled

		Parameter Name	Default Value	Parameter Values
	Energy Protocol	Adult Shock 1 Energy	120 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J, 150 J, 200 J

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
		Adult Shock 2 Energy	150 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J, 150 J, 200 J
		Adult Shock 3 Energy	200 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J, 150 J, 200 J
		Pediatric Shock 1 Energy	50 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J,150 J, 200 J
		Pediatric Shock 2 Energy	70 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J,150 J, 200 J
		Pediatric Shock 3 Energy	85 J	1 J, 2 J, 3 J, 4 J, 5 J, 6 J, 7 J, 8 J, 9 J, 10 J, 15 J, 20 J, 30 J, 50 J, 70 J 85 J, 100 J, 120 J,150 J, 200 J

		Parameter Name	Default Value	Parameter Values
	AED Mode	Begin with CPR	No	No, Yes
		Manual Mode Restriction	User Conformation	User Conformation, Passcode
		Manual Mode Passcode	<blank>	Numeric
		NIBP Monitoring	Disable	Disable, Enable
		CO2 Monitoring	Disable	Disable, Enable
		SpO2 Monitoring	Disable	Disable, Enable
		Temperature Monitoring	Disable	Disable, Enable
		12-Lead Monitoring	Disable	Disable, Enable
		Allow User to End CPR to Analyze	Yes	No, Yes

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
CPR		AHA/ERC Guidelines	2010	2005, 2010, 2015
		CPR Time	2 min	1 min, 1.5 min, 2 min, 2.5 min, 3 min, 3.5 min, 4 min
		Enable Metronome	Always	Never, Always, AED Mode Only
		Constant Metronome	Disabled	Enabled, Disabled
		Enable CPR Prompts	Text and Voice	Text and Voice, Text Only
		Display "Fully Release" Prompt	Yes	No, Yes
		Log CPR Metrics	Yes	No, Yes
		Auto Display Compression Time	No	No, Yes
		Display Feedback Indicators	Yes	No, Yes
		ResQCPR	Disable	Disable, Enable

		Parameter Name	Default Value	Parameter Values
Communications		Wireless	Enabled	Disabled, Enabled
		Bluetooth	Disabled	Disabled, Enabled
		Wi-Fi	Disabled	Disabled, Enabled
		Power Management	Enabled	Disabled, Enabled
		Power Management	Disabled	Disabled, Enabled
		Cellular	Configure	
		Ethernet	Enabled	Disabled, Enabled
		12-Lead Reports	Configure	
		Disclosure Log	Configure	
		Data Services	Configure	
		Clock Sync	Configure	
		Mobile Streaming	Disabled	Disabled, Enabled
Service	Device Tests	Keypad Test LED Test RFU Indicator Test Audio Test Printer Test Battery Test Fan Test NIBP Test USB Test		

APPENDIX A Configuration Tables

		Parameter Name	Default Value	Parameter Values
	Device Configuration	N/A		
	Update/Upgrade	Field Upgrades		
		Software Updates		
	Diagnostic-ECG	General>Resp Breath Marker	Off	On, Off
		Error Counts>ECG CRC Count		
	Diagnostic-NIBP	General		
		Calibration Info		
		Reading Counts		
		Cycle/Error Counts		
		NIBP Test		
	Diagnostic-CO2			
	Diagnostic-SpO2	Version Info		
		Error Counts		
	Diagnostic-Temp/IBP	Error Counts		
	Diagnostic-Defib/Pacer	Version Info		
		Cycle Counts		
		Error Counts		
	Diagnostic-Communications	Operational Log Transfer to USB		

		Parameter Name	Default Value	Parameter Values
	Device Monitoring			
	Research Options	Waveform Recording Sample Rate	250 Hz	250 Hz, 500 Hz
	Supervisor Code	Supervisor Code	<blank>	Numeric
	Import Configuration			
	Export Configuration			