

## Process for Cleaning the ZOLL R Series

The following information details the currently approved process for cleaning the ZOLL R Series device:

- Use only the recommended cleaning agents listed below to clean the device. Follow the directions on the product label for use and storage.
- DO NOT allow cleaning agent or water to run into the crevices or connector openings at any time. Thoroughly wipe off any excess cleaning solution from the R Series unit with a dry cloth.
- Always check monitor and connector opening for unusual wear, damage or dampness while cleaning.
- Cleaning NIBP Cuffs - You can clean the hose and reusable cuffs by wiping the surface with a damp cloth and disinfecting with a mild disinfectant solution. Blow dry air through the hose before use; likewise, ensure the reusable cuff is dry before use.
- Cleaning SpO<sub>2</sub> Sensors - 1. Disconnect the sensor from the patient cable (if attached). 2. Disconnect the cable from the rear of the unit. 3. Wipe the cable clean with a 70% isopropyl alcohol pad. 4. Allow the cable to dry before returning it to use.
- Cleaning Cables and Accessories - Cables, cuff tubing, paddles and other accessories can be wiped clean with the recommended cleaning agents listed below.  
*CAUTION! If bleach solutions or wipes are used to clean ZOLL accessory cables the cable jackets may become discolored. This will not affect the cables function.*
- Cleaning the Print Head - To clean the recorder print head, perform the following steps: 1. Open the printer door and remove any paper. 2. Locate the print head along the front floor of the printer compartment. 3. Gently wipe the print head with a cotton swab moistened with Isopropyl alcohol, and dry any residual alcohol with another dry cotton swab. 4. Place the paper back into the unit and close the cover.

## Products for Cleaning the ZOLL R Series

The following products are recommended for cleaning the ZOLL R Series device:

- Sodium Hypochlorite (Bleach) Solution or Wipes (up to 10,000 ppm)
- Clinell® Universal Wipes
- Ethanol Disinfection with detergent
- Super Sani-Cloth® Wipes
- Sani-Cloth Plus® Wipes
- Oxivir® TB Wipes
- CaviWipes® XL Wipes
- Clorox Healthcare Hydrogen Peroxide Wipes
- 70% Isopropyl Alcohol (IPA) Wipes
- Hydrogen Peroxide Solution
- Wex-Cide® 128 Solution
- Coverage Spray HB Plus
- Warm Water and Soap

